

# Instructions Manual

## Beautyinstrument

WKB2404



VERSION 1.26

## CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
  - Please note that any misuse of the device entails the loss of the guarantee.
  - Do not make any modification to the device without the authorisation
  - we shall not be held liable for any material or personal damage due to the misuse of the product.

## CONTRAINDICATIONS

- This product should not be used on:
  - People with metal implants in their body.
  - People with pacemakers or heart problems.
  - Pregnant women.
  - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

## PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

## SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device, Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

## WARNING

- The patients with following disease are not suitable to use this function, e.g. heart

disease, infectious disease, neuroticism and body with metallic during cure course.






- Do not make the working conduction and export probes touch together for a long time.
- When the skin care essence is insufficient to moisturize, it should be added in time.
- Do not use sliding roller which can not rotate, and should be replaced in time.
- Prevent spray water or other corrosive liquid on the host during using.
- Do not use round probe and roller probe with damaged surface, rusty or worn, to prevent injury of skin. Be careful to protect the probe, to prevent any scarp by other hard matter.
- The host machine shall be turned off and the attaching plug shall be pulled out if any abnormal condition occurs. You shall ask the agency or specific serviceman to check and maintain. The main case is not allowed to be opened.
- It is forbidden to place or use it at bathroom or wet places.
- It is forbidden to put it together with inflammable and explosive objects.
- Forbidden to put probe into water for cleaning and wipe the plastic parts of the probe with alcohol which may damage the probe.

#### Attention

#### Notice

- The zwitterions conduction instrument is normally used after cleaning of skin and before the massage.  
Before treatment, the face should be cleaned, dry, as to prevent bringing any residual cleaning agent into skin basement.
- The probe is rolled tightly with cotton pellet, and the current is adjusted from weak to strong for gradual adaption, then to keep always a light stinging sensation.
- During the treatment course, the hands of the beautician DO NOT touch any metal part of probe.
- The conduction is applicable to dry skin (insufficient water), decrepitude skin (insufficient nutrition) and oily skin. The different liquid beauty products could be selected according to different skin types, but the beauty product for oily skin will prevent permeating.
- After operation, clean the roller, metal bar, and keep in dry place.
- When the polar is set up for positive electrode, the stinging sensation of roller probe will be felt stronger than set up for negative electrode and weaker on the contrary. Please select as your demand.
- During the treatment course, it is not allowed to wear any metal ornament or jewelry, and the client with gold teeth or metallic contained in the body is not allowed to use.
- The client with sensitive skin, injury, inflamed, or telangiectasia and pregnant women is not allowed to use this instrument.
- The pH of nutrient solution should be appropriately matched with positive and negative electrode of rollers, when the roller is in positive electrode, the conductioned solution should be acidity (+); when the roller is negative electrode, the conductioned solution should be basicity (-), if the pH of used nutrient solution has not indicated, pH test paper could be used for determination.
- During the operation, DO NOT directly contact with metal part of roller or round probe, if contacts by accident, the host will activate auto-protection, should separate the probe at once, and the host will cancel the protection and continues for treatment.
- Pls clean and sterilize the probe after uses. Note: if using alcohol to sterilize only for the metal part of the probe, do not wipe other parts of the probe (especially the plastic parts of the probe), otherwise it will damage the prob

## Packing List

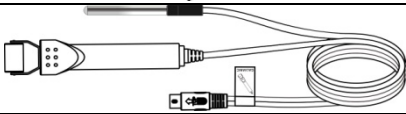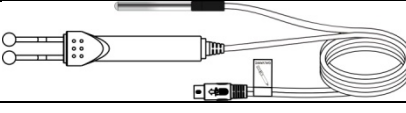
Name	Graphic	Quantity
Host Machine		1 Unit
The power adapter		1 Piece
Roller probe		1 Piece
Round probe		1 Piece
Instruction Manual		1 Copy

## Introduction of Host Machine

1. Power Switch
2. Control Panel
3. Power input plug
4. Galvanic zwitterions probe conduction output




## Accessory

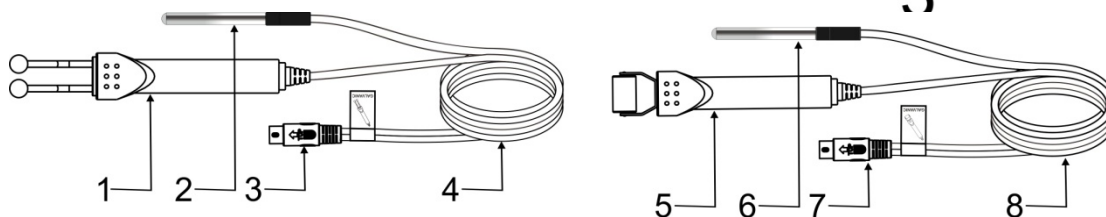

- |   |  |
|---|--|
| 1. Round conduction probe                         | 5. Roller conduction probe                         |
| 2. Sensor   | 6. Sensor  |
| 3. Plug for round conduction probe flexible cable | 7. Plug for roller conduction probe flexible cable |
| 4. Round conduction probe flexible cable          | 8. Roller conduction probe flexible cable          |

## Control Panel

1. Star /pause key
2. Time decrease key
3. Time display window
4. Time increase key
5. Energy increase key
6. Energy display window
7. Energy decrease key
8. Anion conduction icon
9. Cation conduction icon

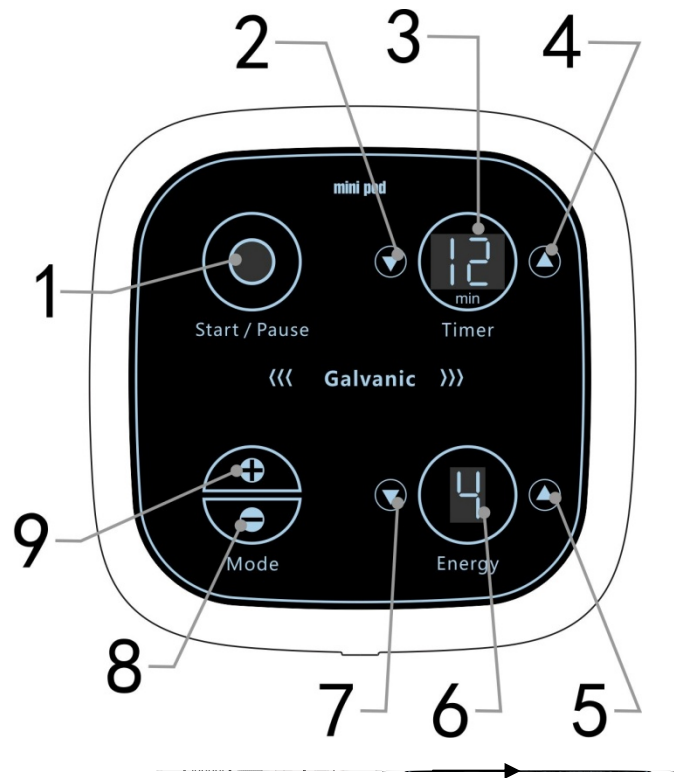

## Control instructions

- Plug the galvanic zwitterions probe 's cord plug into the corresponding output on the back of the main unit. The arrow on the cord plug corresponds to the corresponding position of the main unit's output jack.
- Plug the power plug into the power outlet on the back of the main unit



Turn on the power "🔌" of host machine ,blue indicator light up, host machine is stand-by, would show as below:



Time display "15";  
Energy display "1";


Cation conduction icon  become blue

In this state, you can directly press the "Start/Pause" start pause button to start the host output.

- Function adjustment: In stand by station, press key ,when the light became blue, then the function is good to work(Time adjust and Energy indicate icon would not change)

Time adjust: Press "Timer" increase key  and time decrease key  to adjust host machine working time , time adjust range 1~30 minutes.

Energy adjust: Press "Energy" increase key  decrease key  to adjust energy output, energy output range is 1~9 level.

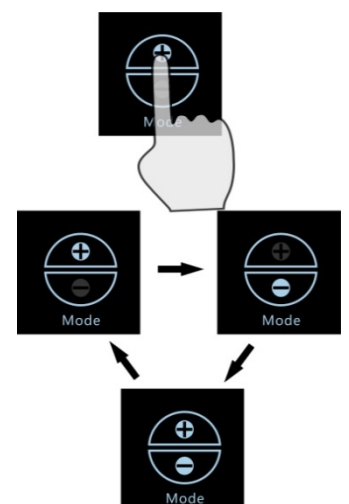


Ion conduction polar selection : “⊕” cation conduction working state; “⊖” anion conduction working state. ,



“⊖” Import the working state alternately by cation and anion, click the icon, and the working mode will be changed by the cycle of cation, anion and anion-cation


**Start and Pause:** In stand by or after adjust press “start / pause” key to start or pause host machine; The startup indicator (Figure 1) will illuminate when the host is started. The point in the lower right corner of the time display window (Figure 2) will flash. Suspend the host, the lights and points will go out.

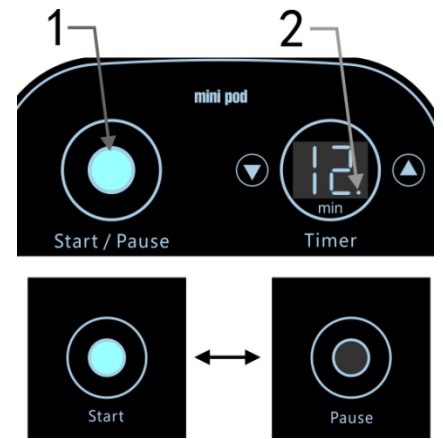

**There are three operation methods based on different treatment effects of zwitterions conduction function**

**A. single-polar conduction method by the beautician**

- Coat directly with half of plant essence on skin, and the other soak the cotton pellet and rolled on the probe.
- Press “⏻”key, turn the energy to min, let customer hold the metal of sensor ,operator hold probe and touch customer treat position ,press “⊖”, host machine would be worked, Press “⊕”or“⊖” to choose positive or negative ion, adjust energy “▲” or“▼” to properly ( Customer would feel a little pricking)。
- The probe moves slowly by a shape of “Z” on face or part of neck with keeping touch of skin during the whole course of duration at 3~5 min.
- After leading ,Press“⊖”to pause host machine , close“⏻”and remove probe, set back sensor 。

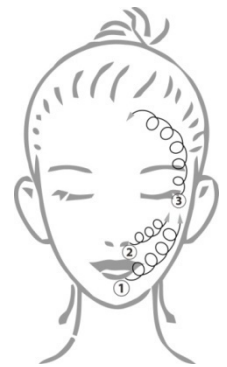

**B. Bipolar Conduction Method**

Impurity is derived when the mobile probe in the work area should be adjusted to the negative, and will be in the customer hand electrode rod anode. The essence, such as impurities, or physiological saline, is soaked in cotton sheets, wrapped around the probe, or covered with a thin layer of cotton or mask that has been soaked through the care of the customer. Then press the ion introduction polarity

selection key to “⊖” “anion” into the working state, the other steps are the same as method 1.。

**C. Alternate output**

The operation mode is the same as method A. In this mode, the probe will alternately output positive current and negative current, conduct physical stimulation on the skin through electrical stimulation, and automatically switch between positive and negative poles every 0.5 seconds.

a

## Daily Maintenance

- The flexible cables for probes and instrument should be in order, Do not repeated twist.
- The instrument should be wiped with dry dishcloth, Do not wet in water.
- After using should be sterilized and wiped dry, and kept in clean, dry.
- Please switch on the main power after using of instrument every day, and takes away the plug when finishes a working day.
- The instrument should not be placed in sunlight, raining, moisture, dust and smoke accumulated area.

## Specification, Model & Technical Data

Product	Beauty Instrument
Model No	WKB2404
Nominal Working Voltage	DC 35V
Rated Power	20 W
Dimension (L*W*H)	18cm×18cm×9cm
Net weight	Kg

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.



Imported by  
WEELKO BARCELONA SL  
Pol. Ind. La Valldan

Cami de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com