


Instructions Manual

Beauty instrument

F801C.N

ULTRASOUND

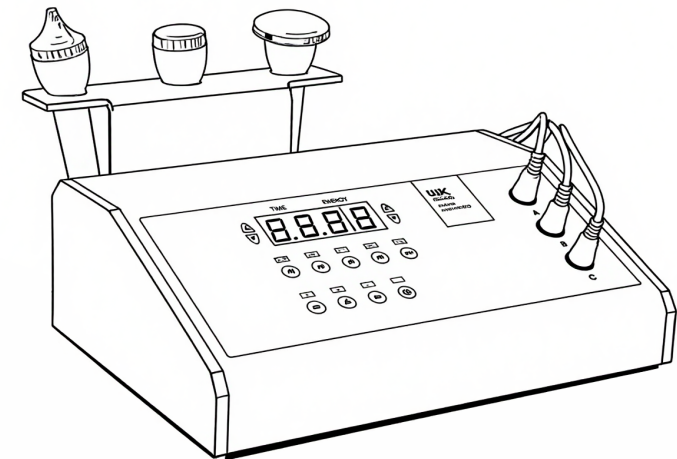

WK
Imported by
WEEKKO BARCELONA SL

Pol. Ind. La Vall d'an
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com

VERSION 1.0

Thank you sincerely for choosing our product. This manual is compiled to help customer service staff use this instrument better. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It is the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instrument's installation and various operation methods.

Usage Precautions

1. Read the user manual carefully before turning on the machine. Operators must be familiar with the user manual and operation methods, and the instrument must be managed by a dedicated person.
2. Do not use a power strip to avoid fire hazards.
3. Always turn off the power switch when replacing the fuse to prevent fire and electric shock.
4. Must use a power supply of 220V±22V, 50HZ±1HZ and an effectively grounded socket.
5. Using a damaged or modified power cord may cause fire or electric shock.
6. If the instrument is loose, falling, or damaged, please contact the company in a timely manner.
7. Do not use the instrument in unstable environments such as tilting, vibration, or impact.
8. No water or debris should enter the interior of the instrument. In case of foreign objects entering, please contact the company promptly.
9. The power plug must be unplugged before moving the instrument.
10. When moving the instrument, try to avoid vibration.
11. The instrument should be placed on a stable surface with at least 5cm of space around it to ensure air circulation and facilitate heat dissipation of the instrument.
12. Do not cover the instrument with anything to prevent blockage of the air vents.
13. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnetic field environments; places with electromagnetic waves or impulse voltage; places where chemicals are stored; places with corrosive gases.

Product Specifications

- Rated Voltage: AC 220V-240V
- Frequency: 50HZ/60HZ
- Maximum Power Consumption: 15W
- Packaging Size: 57x33x17CM
- Gross Weight: 5.8KG
- Net Weight: 4.9KG

Troubleshooting

Fault Phenomenon	Troubleshooting Method
The machine does not work after being powered on	<ol style="list-style-type: none"> 1. Check if the power plug is firmly connected to the power socket. 2. Check if the voltage of the power socket matches the rated voltage of the machine.
No output	<ol style="list-style-type: none"> 1. Check if the machine is in output state (whether the time is flashing). 2. Reinsert the probe firmly.
The probe overheats	<ol style="list-style-type: none"> 1. Turn off the output and let the machine rest for 5-10 minutes.

Product Packaging List

Name	Image	Quantity
Main Unit		1
Ultrasonic Probes (A, B, C)		3
Power Cord		1

14. This instrument shall not be placed or used near radios, televisions, copiers, fax machines, etc.
15. Do not use it together with other instruments (such as microwaves) to avoid mutual interference between instruments and cause misoperation.
16. If an abnormal situation occurs to the instrument or the user during operation, the operator shall immediately move the operating handle away from the user, press the stop switch, and stop using the instrument.
17. The instrument shall be correctly used by skilled operators. Personnel other than operators are not allowed to use the instrument.
18. After operation, place the control handle back on the control handle bracket.
19. Clean the probe of the operating handle after use for next use.

Precautions for Troubleshooting, Storage, and Inspection

1. Do not use the instrument if it malfunctions. In case of smoke, burning smell, or inability to operate, continued use may cause fire or electric shock. Please quickly press the stop button, turn off the power, unplug the power plug, and contact the company or distributor immediately.
2. Personnel other than technical staff are absolutely not allowed to repair or modify the instrument. The interior of the instrument must never be opened.
3. The instrument and its components must be inspected regularly.
4. Do not use cleaning tools that may damage the instrument's protective coating, such as brushes and brooms.
5. If the instrument is not used for more than 2 weeks, it must be carefully inspected before reuse.

Contraindicated Groups

(Do Not Use If You Belong to the Following Groups)

- Pregnant women and lactating women.
- People suffering from severe heart disease, hypertension, or epilepsy.
- People with severe skin inflammation, eczema, dermatitis, sunburn, or unhealed wounds.
- People with cardiac pacemakers or other implanted electronic devices.
- People with a history of skin allergies or allergies to metals, silica gel, or cosmetics.

Post-Usage Care

1. Wipe the skin clean with a cotton pad (especially after extraction).
2. Perform subsequent skin care (such as applying a facial mask or moisturizer) and strictly protect against the sun.
3. Clean the Probe: Clean the probe with a damp cloth and store it after drying as required by the manual.

Safety and Contraindications

Absolutely Prohibited Usage Scenarios:


- Open wounds, damage, severe inflammation, eczema, or sunburn on the face.
- Implanted electronic medical devices such as cardiac pacemakers in the body, or metal implants on the face.
- Pregnant and lactating women (safety not confirmed).
- People with severe diseases such as severe heart disease, epilepsy, or cancer.

Important Precautions:

- Avoid Dangerous Areas: Do not use on sensitive areas such as the eyeballs, eyelids, throat, or thyroid gland (front of the neck).
- Media is Key: Must use water-based media recommended in the manual; do not use oils, scrubs, or products with unclear ingredients.
- Stop if Discomfort Occurs: If you feel any stinging, burning, or other discomfort during use, stop immediately and consult a professional.

Maintenance and Storage

- Clean the probe with a damp cloth and dry it promptly after use.
- Avoid dry burning of the probe; it is recommended to unplug the probe for storage when not in use.
- Store in a dry and cool place, avoiding heavy falling or impact.

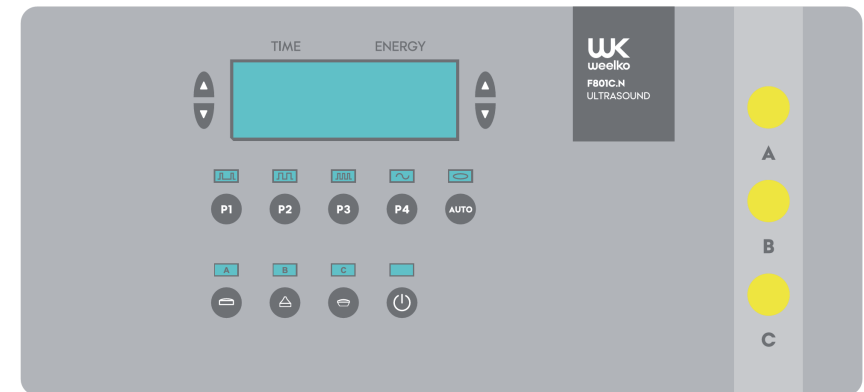

Operation Instructions

1. Plug in the power cord and turn on the power switch. Insert the three probes A, B, and C into the corresponding output sockets.
2. The TIME display shows 15 minutes when turned on, which can be adjusted to 30 minutes.
3. Press the Start/Stop button to begin operation.
4. The ENERGY display shows 0 when turned on, with a total of 10 gears.
5. Press buttons A, B, or C to select the working ultrasonic probe.
6. Press P1-P4 to select programs (Note: Buttons are invalid in AUTO mode).
7. Press the AUTO button (turns on for odd presses and off for even presses). When the light is on, programs 1 to 4 will cycle automatically.
8. After the working time ends, the machine will emit a long beep and turn off all outputs.