0476

RI.MOS. s.r.l. Via Manuzio, 15 – 41037 MIRANDOLA (MO) ITALY – tel. +39 0535 – 25755 –  
fax +39 0535 – 25509 e-mail: info@rimos.com – http://www.rimos.com

**FERTIRAM®**

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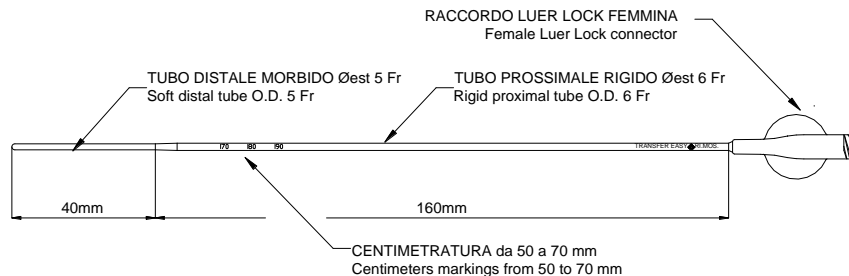
**GB Instructions for use**

Introduce the speculum, expose the cervix and if possible, gently remove the cervical mucus. Lightly swab the uterine cervix with a gauze soaked in embryo culture medium. Connect the catheter to the insuline syringe and introduce the catheter previously loaded with embryos into the uterine cavity. Release the embryos in the preselected point without touching the fundus. Graduation marks on the catheter aid correct placement of the embryos and echogenic tip allows ultrasound-guided embryo transfer. Once embryos have been unloaded, slowly withdraw the catheter and check it under stereomicroscope for any retained embryo.

**Caution:** Read the instructions before using the device. Do not reuse. Use only if the package is not damaged. Use immediately after opening. Dispose after use according to the laws in force. For use only under trained medical staff.

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FERTIRAM products are manufactured from biocompatible materials suitable for human assisted fertilization, MEA & LAL tested and Beta Sterilized.

<b>LOT</b> BATCH CODE	<b>REF</b> REFERENCE NUMBER	ATTENTION. SEE INSTRUCTIONS FOR USE	LATEX FREE	<b>STERILE R</b> STERILIZED BY IRRADIATION
USE BY MM/YYYY	CONSULT INSTRUCTIONS FOR USE	MANUFACTURER	DATE OF MANUFACTURE	DO NOT REUSE

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