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**IDC/F, IDC/GAF: PMC 1 3 CE NO PROTECTION AGAINST MECHANICAL IMPACTS**

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## User Information & User Manual PAULSON IDC Product Line IDC/F, IDC/GAF

### 1. Introduction, Field of Application

PAULSON IDC products are disposable Personal Protective Equipment (PPE) that protects against droplets and splashes of liquids, if used properly. This reduces the risk of a droplet infection. Due to its design, full protection against viruses, bacteria or other biological agents is only given to a limited extent. Can be combined with face masks. These products are supplied in a resealable packaging and are not sterile. Discard after use.

The Declaration of Conformity (DoC) of these products you will find on our website <https://www.paulson-international.com>. Just enter the part number as shown above into the search field and you will be directly transferred to the corresponding document.

### 2. General Warnings

- PAULSON IDC PRODUCTS ARE NOT INTENDED TO PROTECT AGAINST ANY KIND OF MECHANICAL IMPACT !
- Only use as a kit
- Insufficient packing may decrease the level of protection. Thus, follow the instructions as per Chapter 4
- Material which may come into contact with the wearer's skin could cause allergic reactions to sensitive individuals. (REMARK: PAULSON carefully selects the materials and is not aware of any allergic reaction so far.)

Do not use the PPE, if:

- is torn, scratched, cracked or damaged in any way
- is altered from its original form
- has been exposed to chemicals, oils or lubes\*
- has been exposed to high heat or over exposed to sun light\*

\* Damages caused by the aforementioned incidents may not be visible to the end user.

### 3. Usage

#### (a) IDC/F

Wrap the flexible window over the foam strip for easy transportation.

Unwrap it for use. Put on the faceshield by pulling the elastomeric head band around your head upon entry to the patient room or care area. Make sure that no contaminated part from garments or other PPE, such as respirator masks are underneath the IDC/F. When combined with other PPE carefully follow the instructions given by the other PPE manufacturers. Remove the PAULSON IDC product before leaving the patient room or care area and discard it.

DISPOSE AFTER USE!

#### (b) IDC/GAF

The IDC/GAF comes with a specific anti-fog coating, which prevents the goggle from early fogging up. Tear off the protective foil from the goggle surface and put on the goggle by pulling the elastomeric head band around your head upon entry to the patient room or care area. Make sure that no contaminated part from garments or other PPE, such as respirator masks are underneath the IDC/F and the goggle sits comfortably over your eyes. Even your hairs may "bridge" the sealing properties of the foam frame. When combined with other PPE carefully follow the instructions given by the other PPE manufacturers. Remove the PAULSON IDC product before leaving the patient room or care area and discard it.

DISPOSE AFTER USE!

### 4. Storing

For best results store the PAULSON IDC products in a protective bag as used by PAULSON. (NOTE: Various other packing materials are containing solvents which may affect the IDC product's properties.) Store PAULSON IDC products out of sunlight or artificial UV light in a clean, dry environment. The storage temperature shall be between 0°C and 35°C.

### 5. Norms and CE Marking

Paulson IDC products are certified according to EN166:2001:

**IDC/F: PMC 1 3 CE NO PROTECTION AGAINST MECHANICAL IMPACTS**

**PMC** Manufacturer code (PAULSON Manufacturing Corporation)

**1** optical class 1

**3** Protection against splashes of liquids

**CE** CE as per PPE Regulation (EU) 2016/425, Article 17 required for Cat. II products..

### 6. Important Addresses

Manufacturer	European Contact	Notified Body
Paulson Manufacturing Corporation 46752 Rainbow Canyon Road CA 92592 Temecula United States of America Phone: +1 951 676-2451 Email: <a href="mailto:info@paulsonmfg.com">info@paulsonmfg.com</a> Web: <a href="http://www.paulsonmfg.com">www.paulsonmfg.com</a>	PAULSON INTERNATIONAL LTD. Max-Holder-Str. 23 60437 Frankfurt/Main Germany Phone: +49 (0) 69 90 50 585-0 Fax: +49 (0) 69 90 50 585-20 Email: <a href="mailto:info@paulson-international.com">info@paulson-international.com</a> Web: <a href="http://www.paulson-international.com">www.paulson-international.com</a>	ECS GmbH Hüttfeldstr. 50 73430 Aalen Germany Phone: +49 7361 9757396 Fax: +49 7361 5562434 Email: <a href="mailto:info@ecs-eyesafe.de">info@ecs-eyesafe.de</a> Web: <a href="http://www.ecs-eyesafe.de">www.ecs-eyesafe.de</a>

### 7. Hazards Assessment

In order to select appropriate eye and face protecting devices, a hazard and risk analysis shall be carried out by well-trained experts and the eye, face and/or head protector never shall be used in environments with a higher risk rating than the protection rating indicated on the marking. According to this Hazard Risk Assessment, PAULSON IDC products shall be combined with further IDC PPE such as IDC overalls, respirator masks, gloves, etc.

### 8. REACH Declaration

The manufacturer ensures that the products described here and their packaging contain no substances to be included in accordance with Article 57 of the REACH Regulation (EC) 1907/2006 with a concentration of more than 0.1 mass percent (w/w), pursuant to Article 33 of this Regulation.

### 9. Disposal

After use, the PAULSON IDC products are to discard as "contaminated, mixed plastic products".

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**PMC** Código do fabricante (PAULSON Manufacturing Corporation)