

Instructions Manual

Beautyinstrument

WKB2104



VERSION 1.26

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
 - Please note that any misuse of the device entails the loss of the guarantee.
 - Do not make any modification to the device without the authorisation
 - we shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINDICATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device, Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

WARNING

- NOT allow to be no operation under input state of probe, otherwise it will cause





overheat of probe and rapidly reduce the using life.

- NOT try to separate the stainless steel probe and handle, if often rotate the stainless steel probe and handle may damage the probe.
- Forbidden to put probe into water for cleaning and wipe the plastic parts of the probe with alcohol which may damage the probe.
- Forbidden using probe with damaged surface or serious scrape, it must be replaced in this case.
- If there's any unusual circumstances happen, please contact to sales agent and invite professional service people to check, forbid to open host machine and repaired unauthorized.
- It is forbidden to place or use it at bathroom of wet places.
- It is forbidden to put it together with inflammable and explosive objects.

Notice for Ultrasonic Wave Operation:

- The probe should be kept clean; clean and sterilize the probe with ethanol or other agents after uses. Note: if using alcohol to sterilize only for the metal part of the probe, do not wipe other parts of the probe (especially the plastic parts of the probe), otherwise it will damage the probe.
- **DO NOT** use the probe toward eyes when uses ultrasonic wave.
- The skin care essence should be supplemented in time if insufficient to moisturize the skin.
- The nursing time should not more than 15min. and should be shorten appropriately for the small area to be nursed, too long time would cause tired of skin and is not good to absorb. The probe should not stay a long time on one part of skin area to prevent any skin lesion.
- During the course of nursing, it should prevent a long time treatment under the condition of maximum energy output which will not only impact on the absorption of skin but even cause skin lesion.
- The flexible cables for probes and instrument should be in order, **DO NOT** repeated twist.
- If power cord is broken, please use the dedicated cable or from manufacture factory or maintain agency to change. We suggest using accessories from original factory.


Packing List

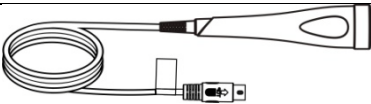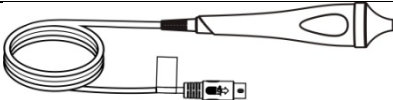Name	Graphic	Quantity
Host Machine		1 Unit
The power adapter		1 Piece
Face probe		1 Piece
Eyes probe		1 Piece




Introduction of Host Machine


1. Power Switch
2. Control Panel
3. Power input plug
4. Probe A (Face) working indicator
5. Probe B (Eyes) working indicator

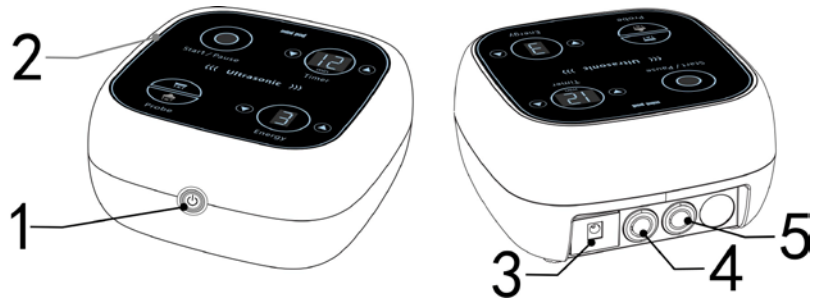

Accessories

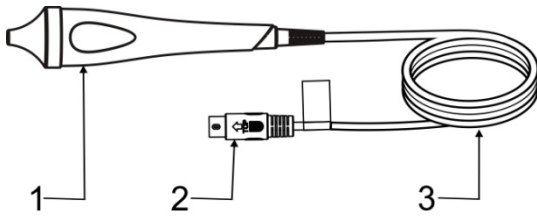


1. Eyes probe
2. Plug for Eyes probe flexible cable
3. Eyes probe flexible cable

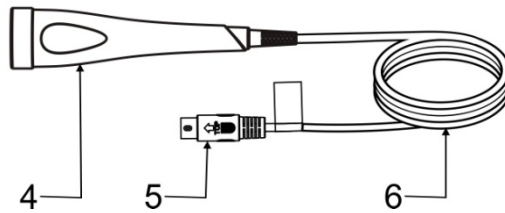

4. Face probe
5. Plug for Face probe flexible cable
6. Face probe flexible cable

Control Panel Introduction


1. Star /pause key
2. Time decrease key
3. Time display window
4. Time increase key
5. Energy increase key
6. Energy display window
7. Energy decrease key
8. Hybrid wave key
9. Continues wave key

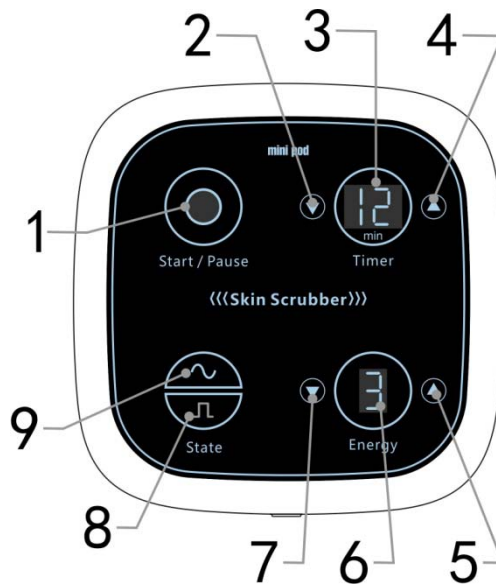


Control instructions

- Plug the ultrasonic wave probe's cord plug into the corresponding output on the back of the main unit. The arrow on the cord plug corresponds to the corresponding position of the main unit's output jack.

- Plug the power plug into the power outlet on the back of the main unit

Turn on the power "  " of host machine ,blue indicator light up, host machine is stand-by, would show as below:

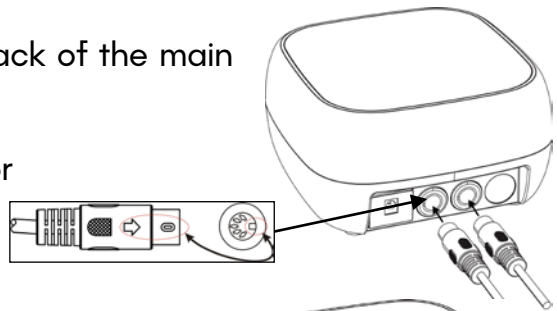



Time display "15";
Energy display "1";



Continues wave key  become blue.

In this state, you can directly press the "Start / Pause" button to start the host output.


- Function adjustment: In stand by station, press key ,when the light became blue, then the function is good to work(Time adjust and Energy indicate icon would not change)

Time adjust: Press "Timer" increase key  and time decrease key  to adjust host machine working time , time adjust range 1~30 minutes.




Energy adjust: Press "Power" increase key  decrease key  to adjust energy output, energy output range is 1~9 level.

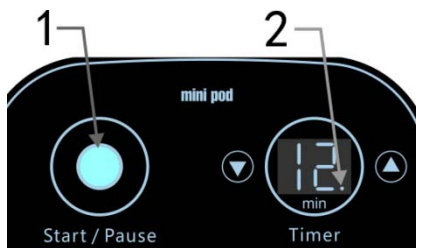

Output probe selection:  is facial probe;  is eye probe.


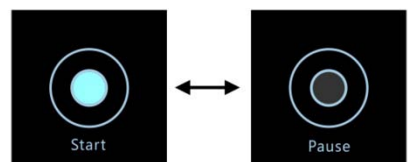


Start and Pause: In stand by or after adjust press "Start / Pause" key to start or pause host machine; The startup indicator (Figure 1) will illuminate when the host is started. The point in the lower right corner of the time display window (Figure 2) will flash. Suspend the host, the lights and points will go ou



Method of application

- Clean facial skin with proper cleansing lotion before use.
- Erase the cleansing lotion.
- Pat each part of the face with astringent water.
- gently erase the astringent water, allowing the skin to be thoroughly sterilized and optimally protected.
- analyze the nature of the skin, select the appropriate ointment and apply it evenly over the face, so that the probe is flexible and rotated
- Choose right probe( is facial probe;  is eye probe)
- Press the power switch, adjust the energy level, in general, do the face can be transferred to the strongest 9, do eye to middle position, 5
- Probe directly treat on skin: the probe moves evenly at the treatment site, or moves in a slow, straight, reciprocating movement for slow, circular movement
- Operate time usually 15 minutes.

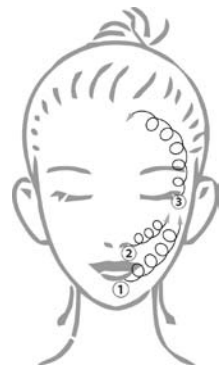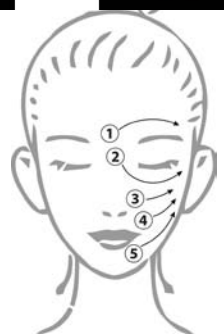

Daily Maintenance:

- The flexible cables for probes and instrument should be in order, **DO NOT** repeated twist.
- Be care of probe during using, and the probe after used should be sterilized and wiped dry, and kept in clean, dry.
- The instrument should be wiped with dry dishcloth, **DO NOT** wet in water.
- Please switch on the main power after using of instrument every day, and takes away the plug when finishes a working day.
- The instrument should **NOT** be placed in sunlight, raining, moisture, dust and smoke accumulated area.

Specification, Model & Technical Data

Product	Ultrasonic wave
Model No.	WKB2104
Voltage	DC 35V
Input Frequency	30 W
Rated power	> 1000000Hz
Size(L*W*H)	18cm×18cm×9cm
Net weight	Kg

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.



Imported by
WEELKO BARCELONA SL
Pol. Ind. La Valldan

Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com