

Instructions Manual

Beautyinstrument

WKB2204



VERSION 1.26

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
 - Please note that any misuse of the device entails the loss of the guarantee.
 - Do not make any modification to the device without the authorisation
 - we shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINDICATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device, Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

WARNING

- **Do not** use water or other corrosive liquid to splashed host machine.

- **Do not** use the deformed and defected scraper knife on the surface stainless, in case of hurt skin.
- The host machine shall be turned off and the attaching plug shall be pulled out if any abnormal condition occurs. You shall ask the agency or specific serviceman to check and maintain. The main case is not allowed to be opened.
- It is forbidden to place or use it at bathroom of wet places.
- It is forbidden to put it together with inflammable and explosive objects.
- Forbidden to put scraper knife. into water for cleaning and wipe the plastic parts of the scraper knife. with alcohol which may damage the scraper knife.


Attention

- Please do not place the working head of scraper knife on skin for a long time, in case of hurt skin.
- Please pay attention to the using strength when doing skin care, place the scraper knife 45 degree with skin is the best (As pictures shown at right side).
- Please supply essence to surface skin to keep moisture.
- Please keep scraper knife 20 degrees with skin (As picture shown at right side) .
- Do not over pull the cable of accessories to the host machine.




Please press the pause key or power key to stop working if not in use, otherwise the long time operation will damage the scrubber and reduce its life time.

- Please take scarper knife gently when using it, and clean it after every time using, but do not put it in the water to clean. After cleaning, please wipe dry and sterilized, keep clean and dry. Note: if using alcohol to sterilize only for the metal part of the probe, do not wipe other parts of the probe (especially the plastic parts of the probe), otherwise it will damage the probe.

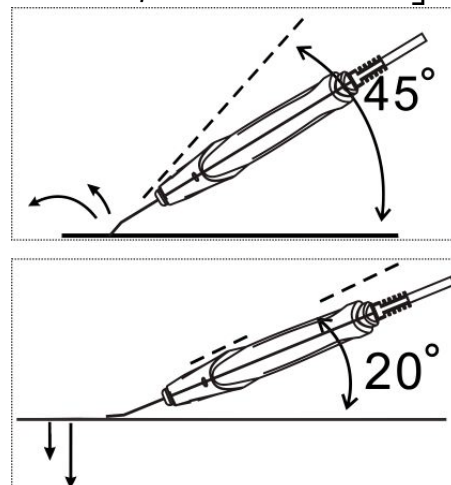- If the cable is destroyed, we suggest using the accessories from repairing agency or original factory.
- NOT allow to be no operation under input state of scraper knife, as this can cause the scraper knife to overheat and rapidly reduce its service life and risk scalding the skin!



Packing list

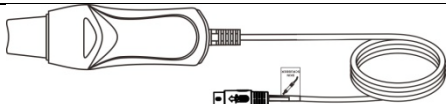Name	Graphic	QTY
Host Machine		1 Unit
The power adapter		1 Piece
Skin Scraper Knife		1 Piece




Introduction of Host Machine

- 1. Power Switch
- 2. Control Panel
- 3. Power input plug
- 4. Skin Scraper




Accessory

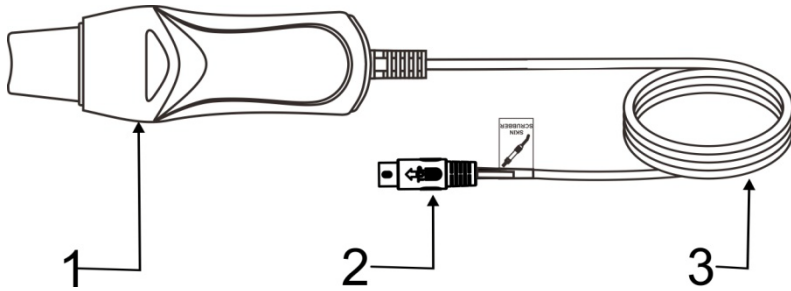

- 1. Skin Scraper Knife
- 2. Scraper Knife Flexible Cable
- 3. Flexible Cable of Scraper

Control Panel Introduction

- 1. Star /pause key
- 2. Time decrease key
- 3. Time display window
- 4. Time increase key
- 5. Energy increase key
- 6. Energy display window
- 7. Energy decrease key
- 8. Hybrid wave key
- 9. Continues wave key

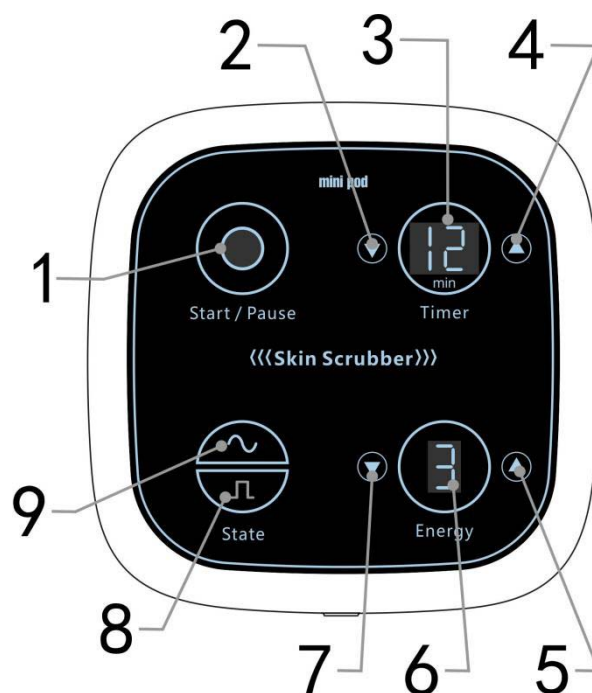

Control instructions

- Plug the skin scraper knife 's cord plug into the corresponding output on the back of the main unit. The arrow on the cord plug corresponds to the corresponding position of the main unit's output jack.
- Plug the power plug into the power outlet on the back of the main unit

Turn on the power "🔌" of host machine ,blue indicator light up, host machine is stand-by, would show as below:



Time display "15";



Energy display "1";



Continue working icon  become blue

In this state, you can directly press the "Start/Pause" start pause button to start the host output.

- Function adjustment: In stand by station, press key ,when the light became blue, then the function is good to work(Time adjust and Energy indicate icon would not change)

Time adjust: Press "Timer" increase key  and time decrease key  to adjust host machine working time , time adjust range 1~30 minutes.

Energy adjust: Press "Energy" increase key  decrease key  to adjust energy output, energy output range is 1~9 level.

Waveform option:  icon is continue wave ;  is wide pulse wave;

Start and Pause: In stand by or after adjust press "start / pause" key to start or pause host machine; The startup indicator (Figure 1) will illuminate when the host is started. The point in the lower right corner of the time display window (Figure 2) will flash. Suspend the host, the lights and points will go out.




Cleaning skin:

- Use sprayer or cleaning water softens skin to be cleaned.
- Press **Energy** key to adjust energy to suitable strength.
- To wipe skin care products on the skin surface evenly (like cleaning milk etc.), in the whole process of cleaning skin care to be kept moist.
- Reference to the presentation of A.B.C.D

A、Eliminate acne without pain(As picture A).Ultrasonic can be split the mature acne into smaller particles in a short time by vibration, and the whole process is painless.

B、Clean-up eruption(As picture A)

With 25,000 times vibration frequency can be cleared the eruption in the pores and residue of cosmetic very quickly.

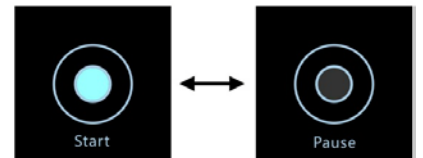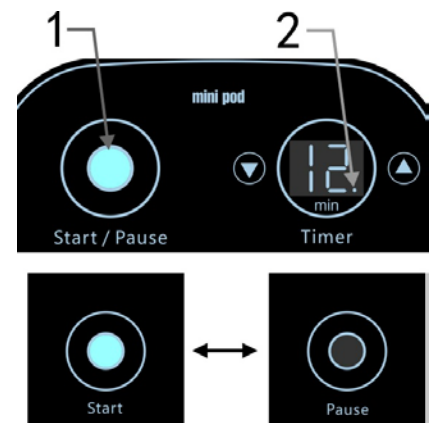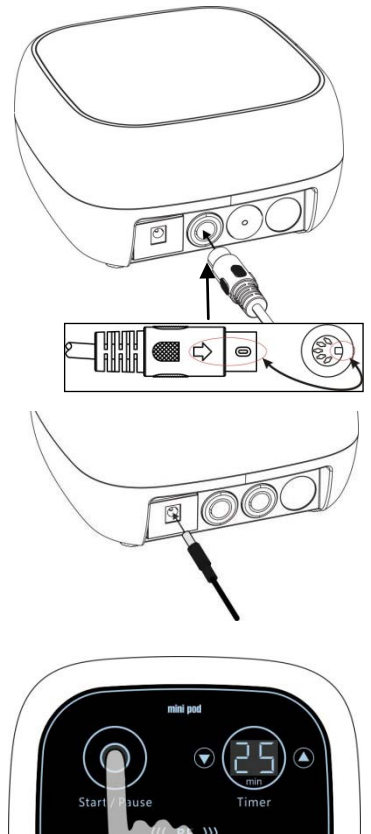

C、Cleaning skin(As picture C)


Scraper knife and be cleaned the dirty things and aging skin.

D、Import massage(As picture D)

Place the head of the scraper knife on the surface skin flatly, and with push-pull can be imported essence to skin effectively(As picture D).

Daily maintain

- The flexible cables for scraper knife should be in order, **DO NOT** repeated twist.
- Please wipe the machine clean with dry cloth, **DO NOT** get it wet.
- Please switch on the main power after using of instrument every day, and takes away the plug when finishes a working day.
- The instrument should **NOT** be placed in sunlight, raining, moisture, dust and smoke accumulated area.

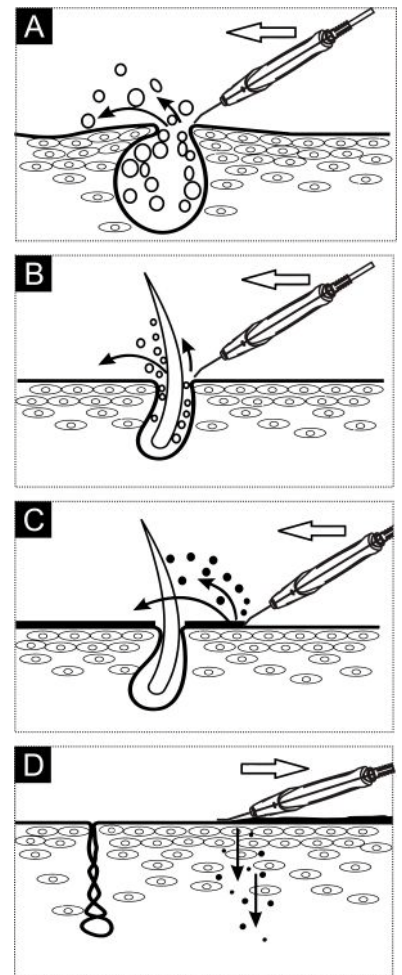

Specification, Model & Technical Data

Product	Skin Scrubber Beauty Instrument
Model No.	WKB2204
Voltage	DC 35V
Rated power	20 W
Output Frequency	25kHz-27kHz
Size(L*W*H)	18cm×18cm×9cm
Net weight	Kg

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.



Imported by
WEELKO BARCELONA SL
Pol. Ind. La Valldan

Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com