

# Instructions Manual

## Beautyinstrument

WKB2304



VERSION 1.26

## CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation
- we shall not be held liable for any material or personal damage due to the misuse of the product.

## CONTRAINDICATIONS

- This product should not be used on:
  - People with metal implants in their body.
  - People with pacemakers or heart problems.
  - Pregnant women.
  - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

## PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

## SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device, Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

## WARNING

- During using of high-frequency electrotherapy, it is not allowed to contact any


metallic, and should be taken off if wearing.

- The client with sensitive skin, patch skin, patient with acne rosacea, wounded skin surface, pregnant women and body equipped with other instrument (e.g. cardiac pacing instrument) **CAN NOT** use this function.
- When using high-frequency electrotherapy, **CAN NOT** use creams and face mask with content of ethanol. If it has to use toner with ethanol content, it can be used after the high-frequency electrotherapy.
- **DO NOT** use each electrotherapy tubes toward eyeball.
- **DO NOT** touch high-frequency electrotherapy main input port or handle input port under condition of live host.
- The plug of accessories flexible cable should be fixed firmly with corresponding outputs of host.
- This instrument should be operated by professional beauticians, because high-frequency interference will be generated during accessories working, **DO NOT** close to other electronic instruments. **BE CAUTION** during using.
- During working, the cables between accessory and host machine shall not be drawn excessively.
- Broken mains cord shall be replaced with specific cord or specific assembly from its manufacturer or maintenance departments. It is strongly recommended that the accessory from original factory shall be used.
- The host machine shall be turned off and the attaching plug shall be pulled out if any abnormal condition occurs. You shall ask the agency or specific serviceman to check and maintain. The main case is not allowed to be opened.
- It is forbidden to place or use it at bathroom or wet places.
- It is forbidden to put it together with inflammable and explosive objects.

#### Attention







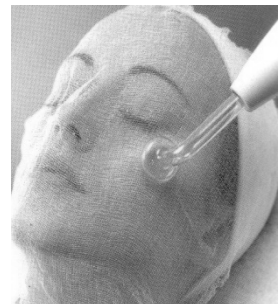

#### Notice for High-Frequency Electrotherapy Operation:

- Squeak sounds during the normal working of electrotherapy handle.
- When direct electrotherapy is used, the spark may spray when the glass electrotherapy tube keeps a distance less than 1 inch with skin, and it would simulate the skin which could make uncomfortable. Therefore, the energy level should be adjusted to the minimum one before contacting with skin. If the spark is felt, a dry cotton yarn can be used to cover the face (Refer to the Fig.)
- The electrode tube should be tightly closed to the face without any gap if direct electrotherapy is available, otherwise, it is easily to generate spark and to stimulate the skin.
- For treatment of face, 8 min. for oily skin while 3~5 min. for dry skin. The current intensity is stronger for oily skin than for dry skin.
- When operation with indirect electrotherapy, the beautician should keep one hand on face of the client, to prevent cut-off of current impacting the effect.
- To prevent electric shock, firstly presses "●" before the glass electrotherapy tube contacts with skin, then slowly increases the energy.
- When takes away the glass electrotherapy tube, firstly adjust the energy to the minimum level, and presses "●" to shut down the output, then take away

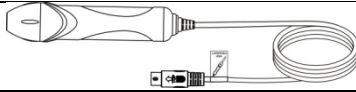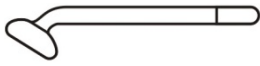

- "🔌" must be turned off when installs and uninstalls the glass electrotherapy tube.
- Each glass electrotherapy tube must be kept clean, dry and sterilized before and after used.
- The electrotherapy handle can be cleaned by towel, **DO NOT** clean in the water, if it is contaminated by oily dirt, it can be cleaned with absolute alcohol, and kept in dry.

### Packing List

Name	Graphic	Quantity
Host Machine		1 Round
Power cable		1 Piece
High-frequency Electrotherapy Handle		1 Piece
Mushroom-shape Glass Electrotherapy Tube		1 Piece
Spoon-shape Glass Electrotherapy tube		1 Piece
Dot Glass Electrotherapy Tube		1 Piece
Straight-Bar Glass Electrotherapy Tube		1 Piece
Instruction Manual		1 Copy


### Introduction of Host Machine

1. Power Switch


2. Control Panel

3. Power input plug

4. High-frequency electrotherapy output

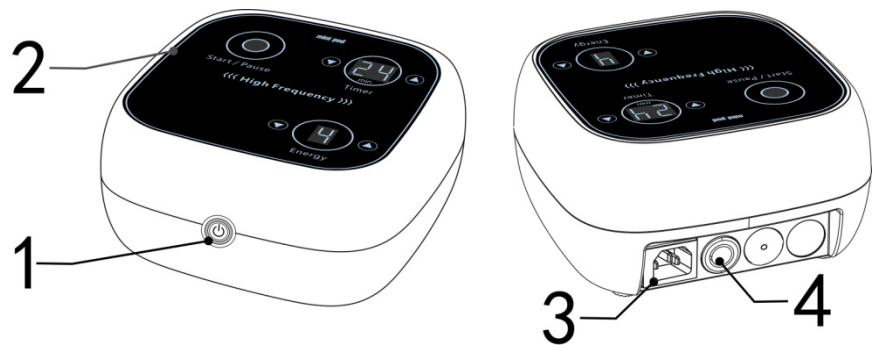


### Accessories

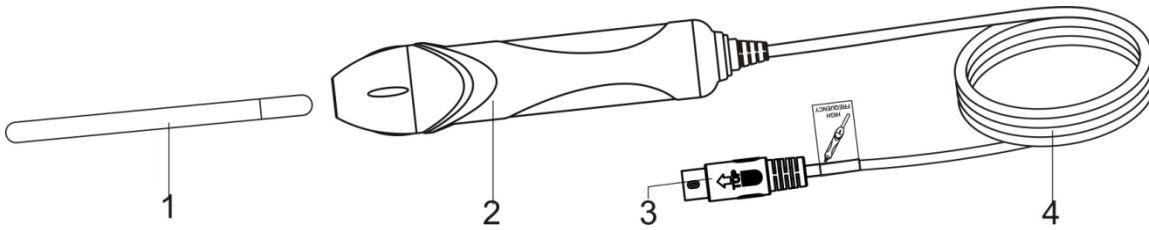

- 1. Electrotherapy Tube
- 2. High-Frequency Electrotherapy Handle
- 3. Plug for Electrotherapy handle flexible cable
- 4. Electrotherapy handle flexible cable


**Different glass electrotherapy tubes are suited for treatment of different parts:**

- A: Mushroom-shape Glass Electrotherapy Tube – large area such as cheek, forehead, neck etc.
- B: Spoon-shape Glass Electrotherapy tube – medium area such as chin etc.
- C: Dot Glass Electrotherapy Tube – small area such as nasal pit etc.
- D: Straight-Bar Glass Electrotherapy Tube – Indirect treatment for clients.

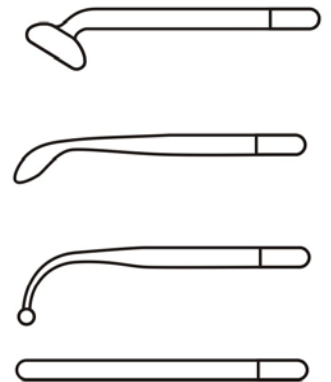


**Assembling and Disassembling of Electrotherapy Tube:**

- 1. Glass part of Electrotherapy tube
  - 2. Metal electrode of Electrotherapy tube
- During the assembling and disassembling of electrotherapy tube, hand should hold the parts within dotted line and keep distance with metal part more than 30mm.

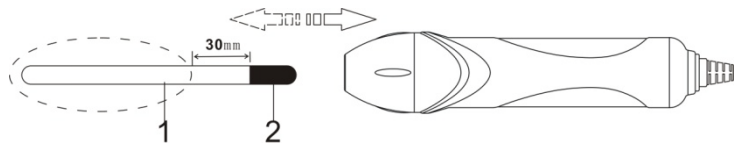

**Control Panel Introduction**

- 1. Star /pause key
- 2. Time decrease key
- 3. Time display window
- 4. Time increase key
- 5. Energy increase key
- 6. Energy display window
- 7. Energy decrease key

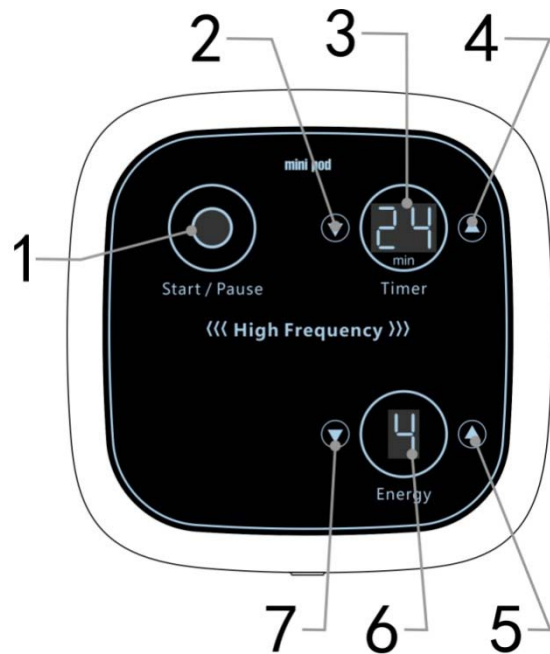

**Operation instruction**

● Plug the High-Frequency Electrotherapy Handle 's cord plug into the corresponding output on the back of the main unit. The arrow on the cord plug corresponds to the corresponding position of the main unit's output jack.

- Connect the power cable with host machine, Turn on the power "🔌" of host machine ,blue indicator light up, host machine is stand-by, would show as below:

Time display "15";  
Energy display "1";

In this state, you can directly press the "Start / Pause" button to start the host output.

- Function adjustment: In stand by station, press key ,when the light became blue, then the function is good to work (Time adjust and Energy indicate icon would not change)

Time adjust: Press "Timer" increase key ▲ and time decrease key ▼ to adjust host machine working time , time adjust range 1~30 minutes.



Energy adjust: Press "Power" increase key ▲ decrease key ▼ to adjust energy output, energy output range is 1~9 level.

Start and Pause: In stand by or after adjust press "Start / Pause" key to start or pause host machine; The startup indicator (Figure 1) will illuminate when the host is started. The point in the lower right corner of the time display window (Figure 2) will flash. Suspend the host, the lights and points will go out.

### Direct Electrotherapy Operate instruction

The beautician holds the handle of high-frequency electrotherapy (see Fig. right), **DO NOT** contact the glass electrotherapy tube with hand or metal matter, switches on the power of host machine, and adjusts "ENERGY" to min. Level.

● contact the glass electrotherapy tube effectively at the customer's treatment site (as shown on the right), then press "🔌" start work and become blue, then press "Energy" key "▲" increase key to adjust output energy, the inner of the glass tube is ionized and emitting light until it has a moderate high frequency current output. In the case of a customer's face care cream or mask, the glass tube is drawn up from the neck of the client and slowly slides to the forehead, face, cheeks, nose, and forehead. In this process, the energy can be slowly adjusted to the required intensity. Because the facial tissue produces heat, direct use of high frequency electrotherapy has a soothing and calming effect. Also has bactericidal effect, suitable for oily and acne skin use.

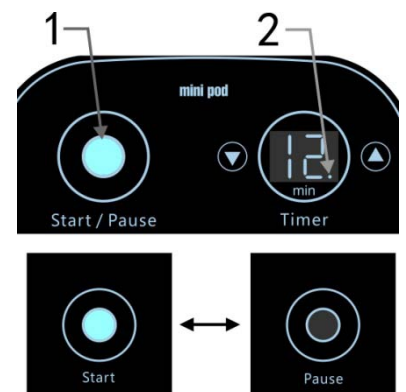
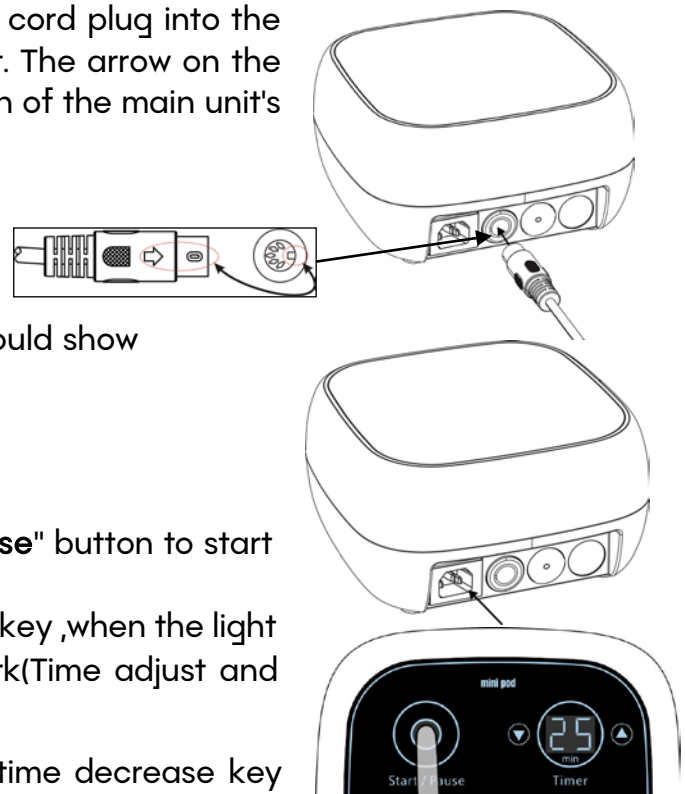


- After treatment, adjust energy to min ,then press “●” pause the output ,then remove the glass electrotherapy tube from customer face



### The Functions of Direct Electrotherapy:

- Warm vibrations have the effect of calming and soothing the skin.
- Promote blood circulation, provide skin nutrition, make skin shiny and elastic.
- Increase cell metabolism, and help the skin breathe and excrete
- The thermal effect on the skin, increase the permeability of the cells, contribute to the enhancement of the skin absorption capacity, and strengthen the effect of nursing.

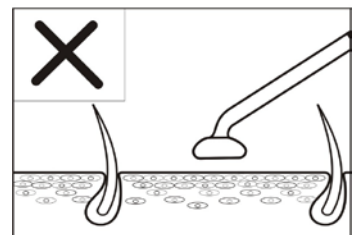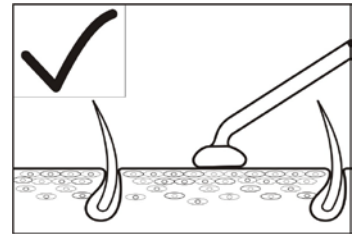

### Indirect electrotherapy:

Let the customer hold the straight stick electrotherapy tube first, then press the key “●”to start, adjust “Energy” to properly , then operator massage customer treatment position. During the process, the operator did not touch the electrode before removing the holder of the glass electrode holder in the customer's hand, Must press“●” key to make host machine pause output.

Function of indirect electrotherapy:

- stimulate blood circulation and increase metabolism.
- adjust and restore the absorption of nutrients by skin tissues and resistance to bacteria.
- Because of the action of the current, it can enhance the vitality of the bottom cells of the skin and improve the massage effect.



### Daily Maintenance:

- Be care of glass electrotherapy tube during using, and carefully sterilizes.
- The tightness of electrotherapy tube should be check frequently, test method: puts the tube to be tested into the handle, and places the handle stably on the table, connects with cable of host, starts up the host, and increases gradually the energy output, it is normal to observe the lighting in electrotherapy tube.
- Periodically wipes the instrument with dry dishcloth and to place in dry, ventilation area, the cable of instrument should be place in order but not twist.
- The instrument should be wiped with dry dishcloth, **DO NOT** wet in water.
- Please switch on the main power after using of instrument every day, and takes away the plug when finishes a working day.
- The instrument should **NOT** be placed in sunlight, raining, moisture, dust and smoke accumulated area.

## Specification, Model & Technical Data

Product	Beauty Instrument	
Model No	WKB2304	
Functions	High-Frequency Electrotherapy	
Nominal Voltage	Working	ac. 100-120V / 220-240V
Nominal Frequency	Working	50/60Hz
Rated power	20 W	

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

**WK**

Imported by  
WEELKO BARCELONA SL  
Pol. Ind. La Valldan

Camí de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com