

Instructions Manual

Beautyinstrument

WKB2504B



VERSION 1.26

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation
- we shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINDICATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS


- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device, Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

WARNING

- Be care of flexible tube connecting accessories and host, **DO NOT** over-pull.

● Prevent spray water or other corrosive liquid on the host during using.


● Forbidden connecting the flexible tube of spray function with output of vacuum function (see the Fig. right) as well as connecting of flexible tube of vacuum function with nozzle of spray bottle (refer to the Fig.), otherwise it will suck back the nursing liquid into spray bottle, and directly damage the host.

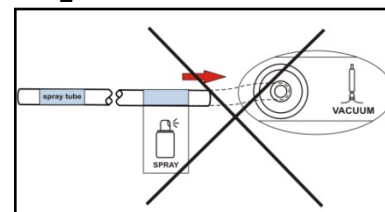

● Forbidden using sucker with edge angle, sharp edge, worn and damaged. In this case, please contact with distributor or appointed maintenance center to replace in time.

● During working, the cables between accessory and host machine shall not be drawn excessively.

● Broken mains cord shall be replaced with specific cord or specific assembly from its manufacturer or maintenance departments. It is strongly recommended that the accessory from original factory shall be used.

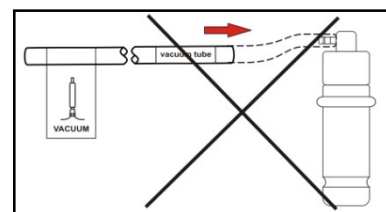

● The host machine shall be turned off and the attaching plug shall be pulled out if any abnormal condition occurs. You shall ask the agency or specific serviceman to check and maintain. The main case is not allowed to be opened.

● It is forbidden to place or use it at bathroom of wet places.

Attention

Notices for Operation of Spray Vacuum Functions:

● Carefully check the label on air flexible tube to prevent incorrect connection.

● The host should be paused if the spray bottle or sucker needs to be replaced during using.

● During the working of vacuum or spray function, **DO NOT** bend the flexible tube or block air inlet, which may directly cause the damage of host.

● Any crack on nozzle of flexible tube will directly impact on the working effect, to cut the crack part with scissor and re-connects with corresponding joint.

● It should be noted during operation of spray function:

>Controls the spray mist volume, to prevent the waste of skin refreshener.

>To prevent water drop to neck, where should be protected with water basin or towel.

>**NOT** spray toward naris or mouth, to guarantee the breath.

>**NOT** fill in spray bottle with liquid with low viscosity, to prevent any block.

>The spray bottle should be cleaned in time with distillate or purified water after each time use, and wipes dry and sterilizes for next time use.

● It should be noted during operation of vacuum function:

>The skin parts should be cleaned and softened before operation of vacuum function.

>The sucking intensity of vacuum tube should be controlled medially, high intension could cause skin lesion and coagulation of blood below the surface of the skin. The beautician firstly could try by own hands, and moves to working parts after adjustment of vacuum ability.

>When the nursing is made on fix parts of skin, the press time of relief-pressure air inlet should be controlled by fingers, to prevent too long time sucking and become purple by sucking.

>The sucking frequency should be increased for oily, thick skin, and sucking is not allowed to be used for skin with telangiectasis.

>**DO NOT** use for eye skin or face with inflammation.


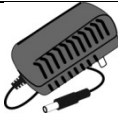

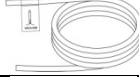

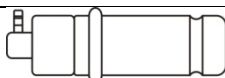
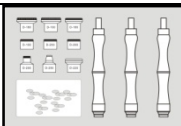
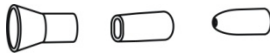


>**DO NOT** use frequency the vacuum sucking, to prevent enlarge of skin pore.

>The vacuum bar should be uninstalled in time after use, and clean, wipe dry and sterilizes for next time use.

>The clean sponge filter should be replaced after each time using, **DO NOT** use vacuum bar without sponge filter, which will directly damage the host.


- Pls clean and sterilize the probe after uses. Note: if using alcohol to sterilize only for the metal part of the probe, do not wipe other parts of the probe (especially the plastic parts of the probe), otherwise it will damage the probe.

Packing List

Name	Graphic	Quantity
Host Machine		1 Unit
The power adapter		1 Piece
Spray hose tube		1 Piece
Suction hose tube		1 Piece
Sponge filter		3 Pieces
Spray bottle		2 Pieces
Diamond probes& handle		1set
Sucker		1 set
Vacuum tube		1 Piece
Instruction Manual		1 Copy


Host Machine

1. Power Switch
2. Control Panel
3. Power input plug
4. Suction function output
5. Spray function output

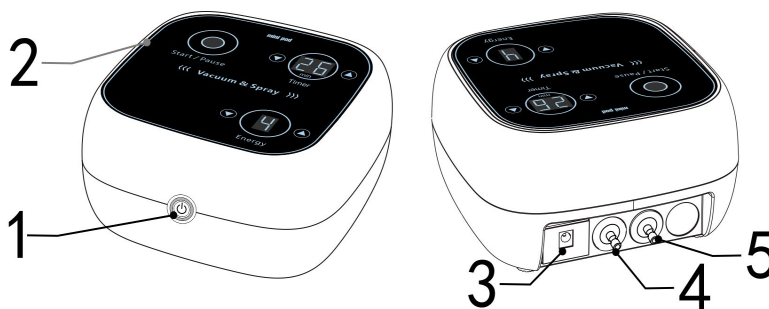


Accessories

1. Spray bottle
2. Spray bottle air flexible tube
3. Horn mouth sucker
4. Flat mouth sucker
5. Round let mouth sucker
6. Suction bar air flexible tube
7. Suction bar

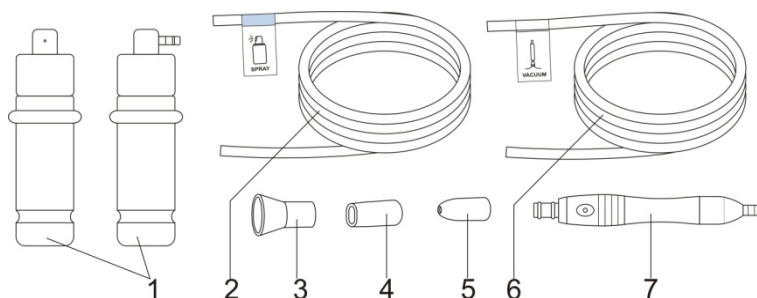


Spray Bottle:

1. Nozzle
2. Cover of bottle
3. Suction tube
4. Spray bottle
5. Air inlet joint
6. Air vent

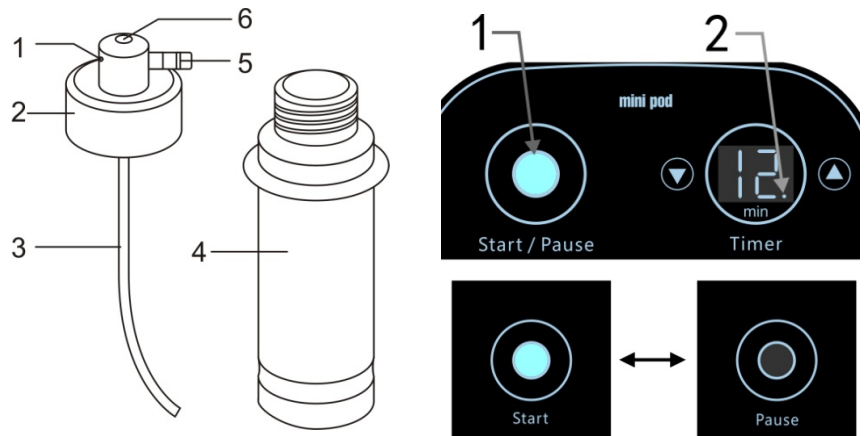


Suction Bar:

- 1 Round let mouth sucker
- 2 Seal ring of sucker
- 3 Relief-pressure air vent
- 4 Suction bar
- 5 Seal ring of bar
- 6 Sponge filter
- 7 Suction bar end

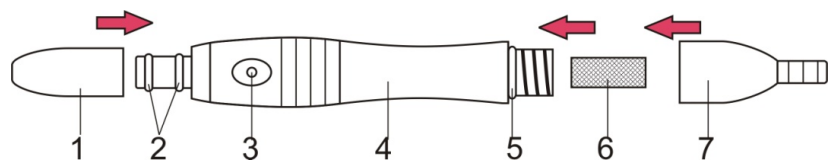


1. Diamond probes
2. Handle

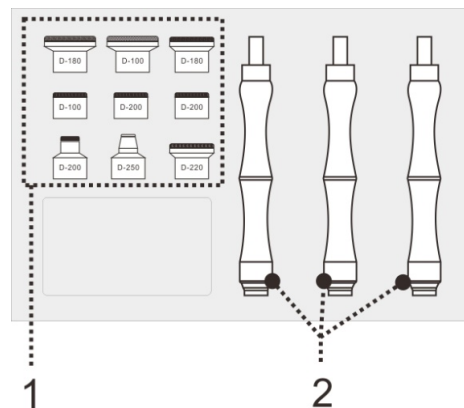


Control Panel Introduction

1. Star / pause key
2. Time decrease key
3. Time display window
4. Time increase key
5. Energy increase key
6. Energy display window
7. Energy decrease key

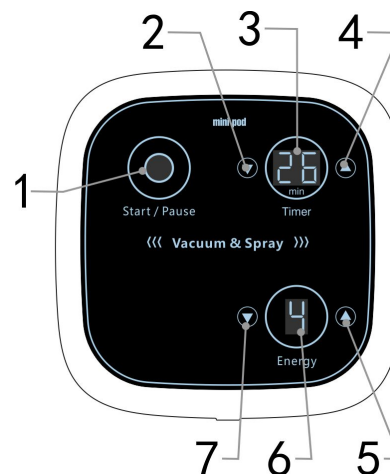


Control Instruction


- please remove the screws which are fix the pump before use (otherwise it will cause abnormal sound when operate), Plug the power plug into the power outlet on the back of the main unit; The air guide hose marked "spray tube" is effectively inserted on the bottle interface, and the other end of the hose is effectively inserted in the spray function output port of the host machine

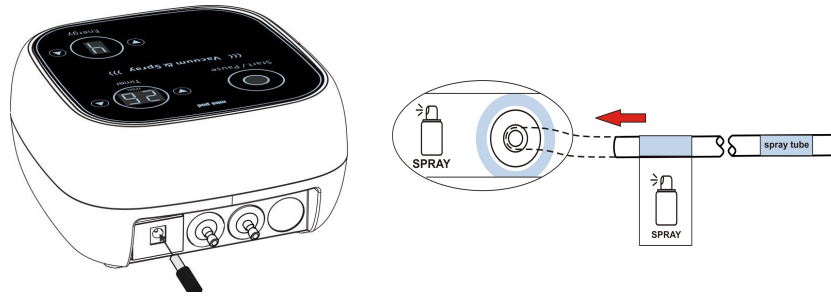

- Air guide hose with "vacuum tube" is effectively inserted on the connector of vacuum holder, other end will be effectively inserted output port of host machine, ; Unscrew the suction rod tail, view the suction rod center whether filter sponge, if not please add cotton filter and screw rod suction; suction rod connected at the front end of the transparent suction head, please refer to the installation, slowly rotates into, Make sure the sealing ring is not deformed so as to keep the air tightness, as shown.

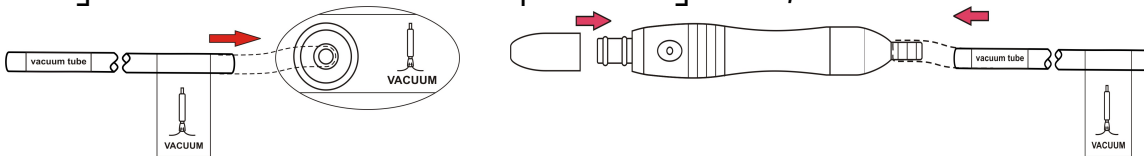

please connects the handle with connection hose labeled with "vacuum tube",The other side connects with the output from the host. Screw one of the diamond probe with the handle. Please keep the shape of the rubber band to be sealed when insert the probe slowly. (as picture shown)

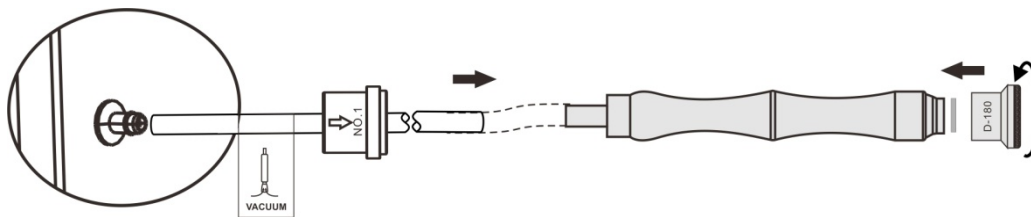

Turn on the power "🔌" of host machine ,blue indicator light up, host machine is stand-by, would show as below:

Time display "15";

Energy display "1";

In this state, you can directly press the "Start/Pause" start pause button to start the host output.



- Function adjustment: In stand by station, press key ,when the light became blue, then the function is good to work(Time adjust and Energy indicate icon would not change)

Time adjust: Press "Timer" increase key ▲ and time decrease key ▼ to adjust host machine working time , time adjust range 1~30 minutes.

Energy adjust: Press "Energy" increase key ▲ decrease key ▼ to adjust energy output, energy output range is 1~9 level.

Start and Pause: In stand by or after adjust press "start / pause" key to start or pause host machine; The startup indicator (Figure 1) will illuminate when the host is started. The point in the lower right corner of the time display window (Figure 2) will flash. Suspend the host, the lights

and points will go out.

Operation Method of Spray Vacuum Functions

Spray Function: Turn on the power of the host, adjust the working time and energy, and click the start key "●".

Under start up of host output, hold tightly the spray bottle by hands, and presses slightly the vent at top of spray bottle by fingers, then the mist will spray from the mouth; two spray bottles are prepared, for different liquids, and the bottle could be switched alternatively based on demand; the host should be pause for output when switch the spray bottle.


It should be careful for nursing of critical parts e.g. Mouth, nose, eyes. Please close the eye, mouth to prevent liquid sucked into the naris and should spray out toward below of chin, and keeps certain distance with face. After spray, pauses the host, and absorbs remained liquid on surface with cotton pellet or soft paper.

Vacuum Function :

Turn on the power of the host, adjust the working time and energy, and click the start key "●".

Under the start up of host, hold tightly the vacuum bar by hands (like holding a pen), moves the bar to working location, the nozzle of transparent sucker should contact with skin, and presses slightly the relief-pressure air vent, then the skin contacting with sucker is absorbed, and moves around the bar on related parts, where allows to absorb black head, acne in skin.

Different suckers should be selected based on the different skin areas:

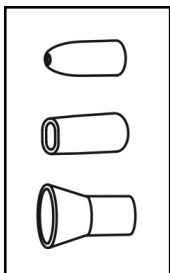

- Round let mouth sucker is used for black head, acne in small area skin;
- Flat mouth sucker is used for black head, acne in narrow area skin;
- Horn mouth sucker is used for black head, acne in big area skin.

MicrodermabrasionFunction :

Turn on the host and adjust the timer and energy, host will keep pause of output.

When the host starts to output, hold up the handle, close the skin with the probe and intake it by negative pressure, move back and forth on the skin.

Please choose certain probe as per different skin area when treatment for clients.

Daily Maintenance:

- Be care of probe during using, and the probe after used should be sterilized and wiped dry, and kept in clean, dry.
- The sucker should be kept clean.
- The nozzle of spray bottle should be kept in clean and unblocked.
- The instrument should be wiped with dry dishcloth, **DO NOT** wet in water.

- Please switch on the main power after using of instrument every day, and takes away the plug when finishes a working day.
- The instrument should NOT be placed in sunlight, raining, moisture, dust and smoke accumulated area.

Specification, Model & Technical Data

Product	Spray Vacuum Microdermabrasion Beauty Instrument
Model No	WKB2504B
Voltage	DC 35V
Rated power	50W
Dimension (L*W*H)	18cm×18cm×9cm
Net weight	2.1Kg

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

WK

Imported by
WEELKO BARCELONA SL
Pol. Ind. La Valldan

Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com