weelko

# Instruction manual

BLight 502 Beauty Chair  
BLight 503 Podiatry Chair

V2

## Table of contents

1. Use of the manual
2. Warranty
3. Product Liability
4. EC Declaration of conformity
5. Safety instructions
6. Product Dimensions
7. Features & Operation
  - Arm rests
  - Breather hole and plug
  - Hand controller
  - Foot controller
  - Extensible parts
  - Legs (BLight 503)
  - LEDs
8. Maintenance and cleaning
9. Troubleshooting
10. EMC Guidance

## 1. Use of the Manual

Before using the chair, please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

By carefully following these instructions you will be able to avoid any problems in using the chair.

Some texts and/or illustrations may not belong to your B-Light chair, since they are general and it depends on your product choice at the purchase time.

Our company reserves the right to improve design and modify the products without prior notice.

## 2. Warranty

The warranty period is effective on the date of purchase of the treatment table. It covers all the components of the beauty bed under the normal and correct use. It also covers defects in workmanship from factory. The above-mentioned warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the bed and its upholstery;
- Non-observance of the precautions and warnings stated in this instruction manual;
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer;
- Wrong and faulty electrical installations or which do not fulfil the legal requirements;
- The company is not liable for incidental or consequential damage to property or business.

The manufacturer will not be held responsible for the results of maintenance or repairs performed by unauthorized persons. For further information on guarantee terms, please contact your distributor.

### 3. Product Liability

Under any circumstances, the producer shall be liable for any indirect, special, incidental or consequential damage incurred by the user, the patient or the bed itself as a result of:

- a. A wrong diagnosis;
- b. Incorrect use of the bed or any of its accessories;
- c. The wrong interpretation or non-observance of this user manual;
- d. A lack of maintenance of the bed;
- e. Or maintenance works of the bed and/or repairs performed by persons unauthorised to do so by the manufacturer.

According to the current legislation, the manufacturer cannot be held responsible for any flaws in the chair or its components and accessories after a period of ten (10) years from the date on which the producer puts into circulation the actual product. Therefore, producer cannot be obliged to supply any spare parts or accessories for this chair.

### Expected lifespan

Provided that the operator and user takes care of the table and undertakes regular maintenance tasks as described in section 8 of the current manual, the estimated lifespan of the bed is 10 years from production date.

The intended use of the chair is to position a patient for the treatment. The unit is intended for use by qualified professional within a professional setting.

### 4. EC Declaration of conformity

Our products are produced and marked in conformity with the directives hereby stated :

Low voltage : 2014/35/CE

Electromagnetic compatibility : 2014/30/CE

These products are manufactured according to the essential requirements established in the European Medical Devices Directive (Annex II+III of Regulation (EU) 2017/745):

EN ISO 14971:2019; EN ISO 204117:2021; EN ISO 15223-1:2016; ISO 10993-1: 2018; EN ISO 10993-5:2009; EN ISO 10993-10:2013; EN 60601-2-52:2010+AC:2011+A1:2015; EN 60601-1-2:2015+A1:2020; EN 60601-1:2006+A1:2013+AC:2014+A12:2014+A2:2020.

### 5. Safety instructions

#### Responsibility of the owner

Our beauty and podiatry chairs are only meant to be used by professional personnel in beauty, aesthetics, podiatry, dermatology and medical aesthetics fields.

It is the responsibility of the owner to ensure that training is given to all operators and third parties (like for example cleaning personnel) on safe use of the chair, and that they have read and understood all the operating and safety instructions.

Only trained operators may use the treatment bed. Please keep the manual in a place where it is easily accessible to all users, and make it an important part of your employee training program.

## **Safety on installation and electricity**

- Place the table in a space with free access from the 4 sides. Do not place the table against the wall or against any obstacles.
- Do not use the table in a high-humidity environment
- Check if the mains voltage and frequency are in accordance with the information on the typeshield. If not, contact your supplier before connecting the device.
- Please pay attention that the continuous working times should be “Max. 2min. ON; 18min. OFF” when operating the motor.
- Make sure that the unit is correctly plugged into an earth intake, following the local and national requirements and standards.
- Make sure that the mains cable is positioned in such a way that it cannot get stuck between the mechanism.
- Especially pay attention that the mains cable does not get pinched under the feet of the table.
- If the table gets wet accidentally, unplug the table immediately.
- Do not leave children unattended near the table.
- Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lack of experience, common sense or children, are not allowed to use the equipment.
- Do not use the device near electromagnetic transmitters. Read and follow all the information about electromagnetic compatibility (EMC) in section 10 of this manual.
- If the power cord or plug is damaged, please contact the distributor, service center or experienced technicians for repairing.

## **Safety during treatments**

- Don't let anybody, except the operator and the patient, in the treatment room.
- The bed is not intended to transport patients. Therefore it cannot be used for this purpose.
- Do not sit on the head section or end of the leg section of the table. The head section and armrests are not suitable to sit on.
- Patients are not to be left unattended.
- Do not exceed the maximum load capacity of the table.
- Place the bed on a stable and horizontal surface. Level it if necessary with the levelling feet under the table.
- Ensure that neither the patient nor the operator can get any part of the body or clothes trapped between the moving parts of the mechanism when the table is being raised or lowered.

## **Safety before or after the treatment**


- Do not leave anybody unattended alone near the bed.
- Disconnect the table if not used for a long period of time.

## **Safety when cleaning the bed or the therapy room**


- Disconnect the table from the power supply.
- Do not get under the bed or between its moving parts.

## 6. Product dimensions


### BLight 502

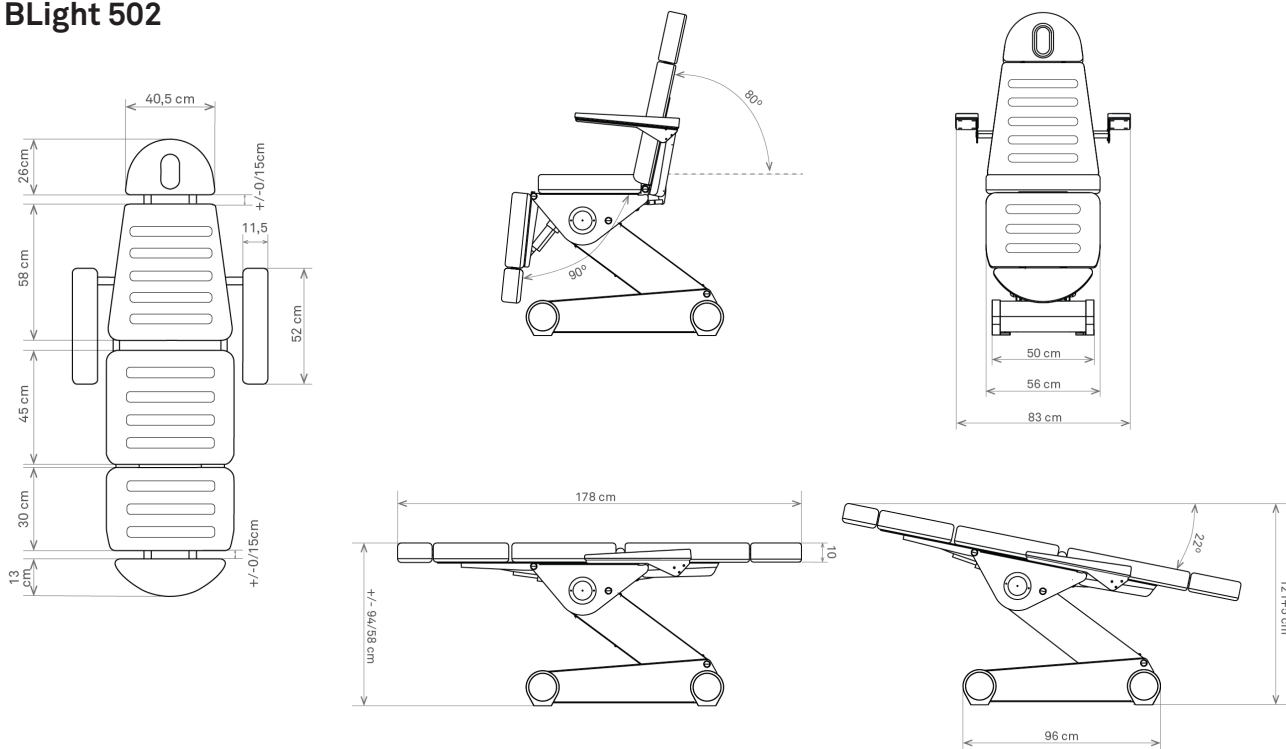
### BLight 503
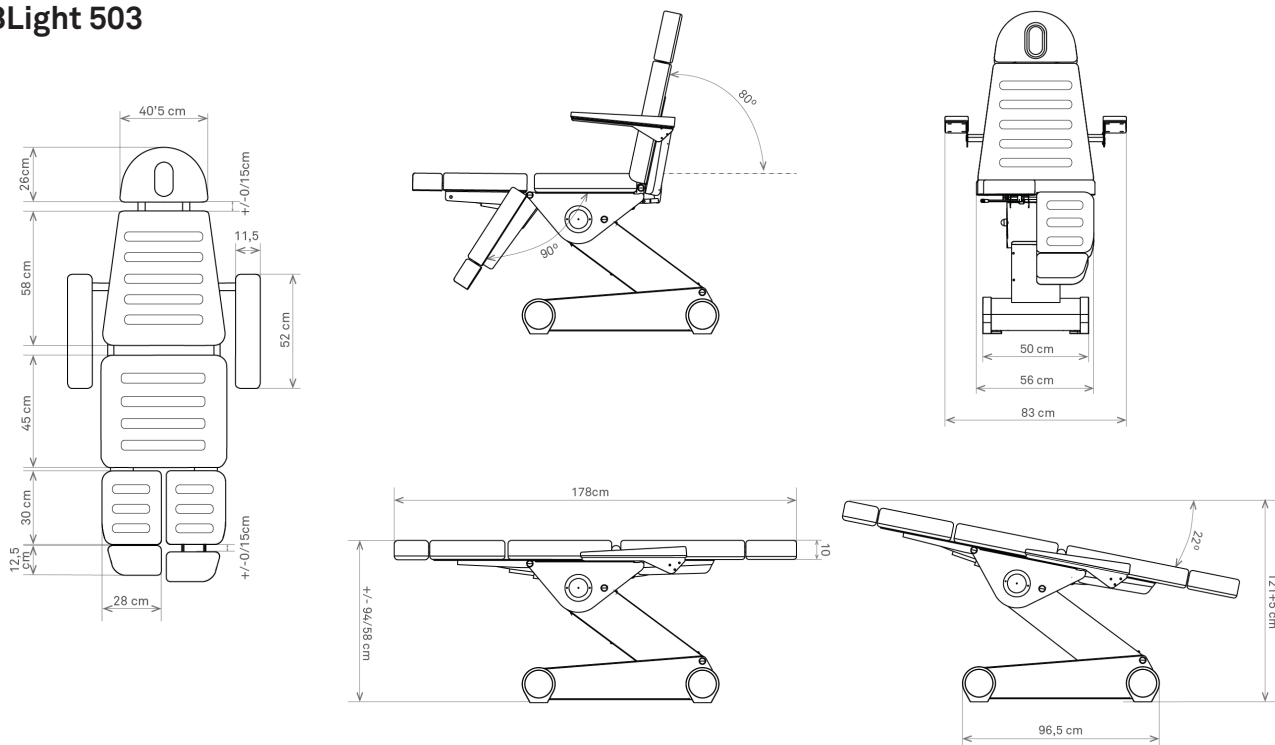
### Product dimensions in movement (cm)

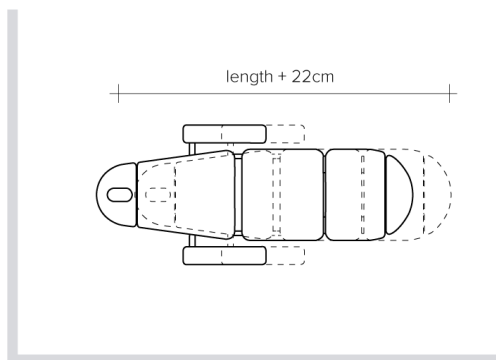
### Power cord length (cm)
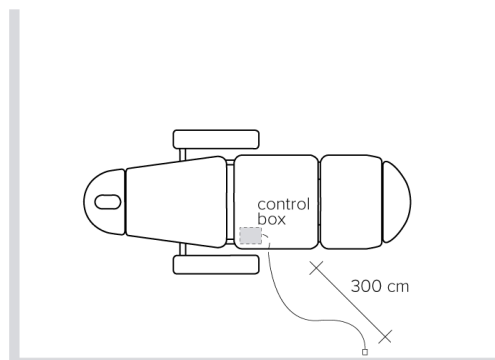
## 7. Operation







### Power input

Voltage and frequency input shall be 100-240V, 50/60Hz. Plug the table cable into the socket as shown below: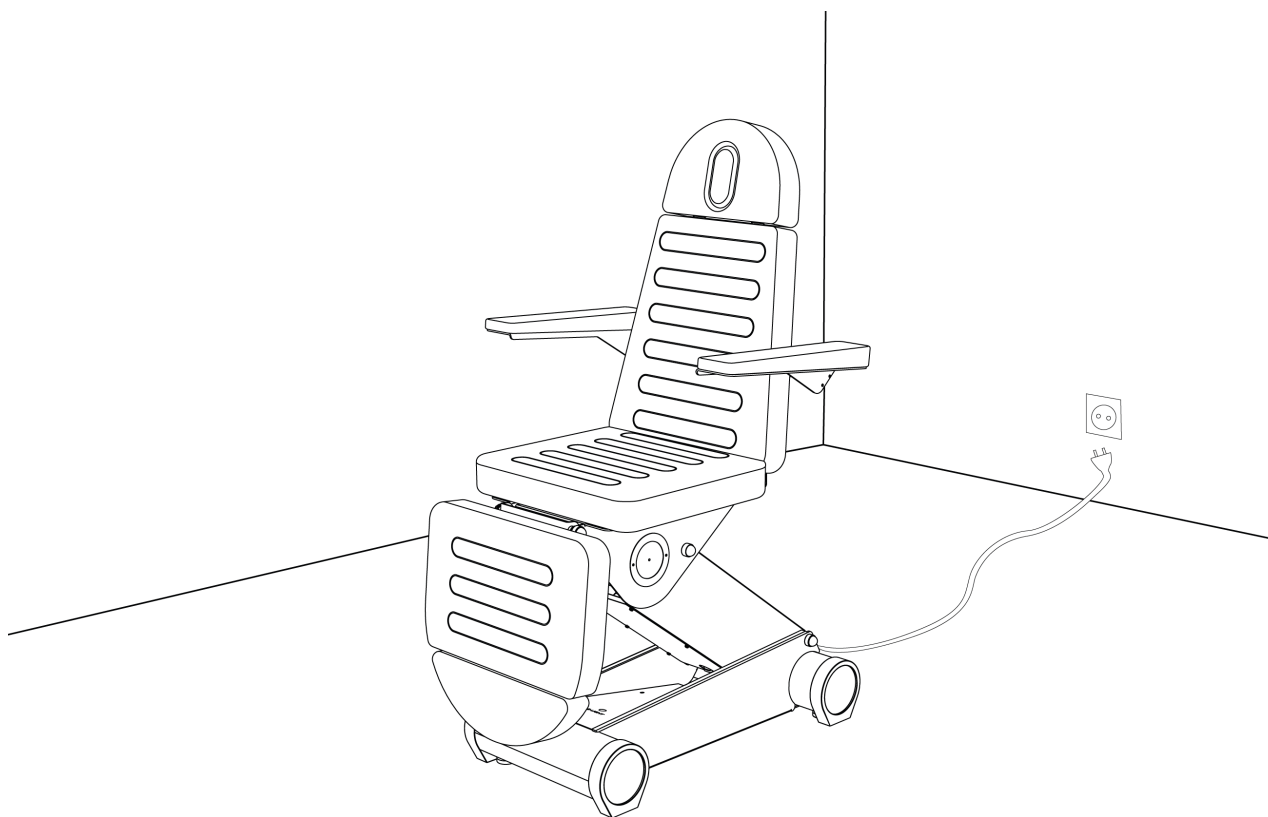

### Operation of BLight 502 and 503

1. Arm rests
2. Breather hole and plug
3. Hand controller
4. Foot Controller
5. Extensible parts
6. Legs (BLight 503)
7. LEDs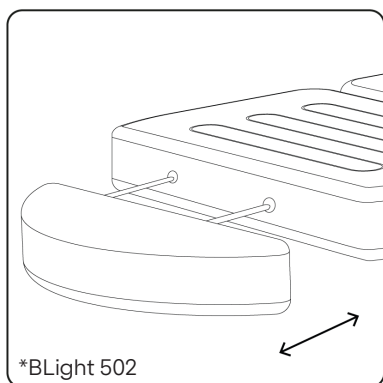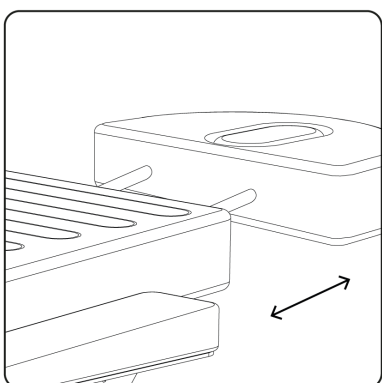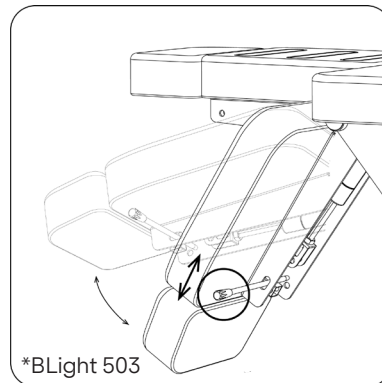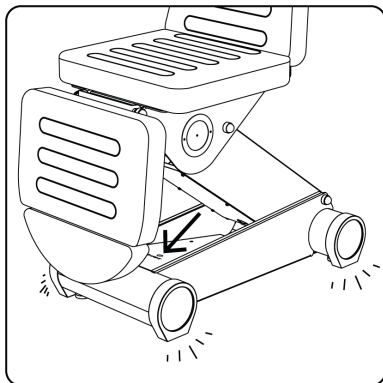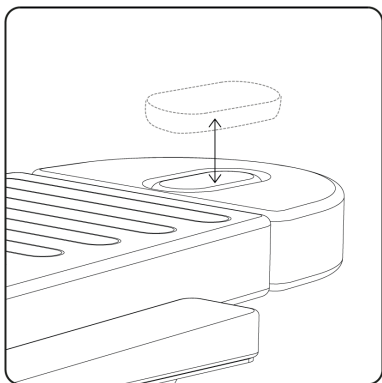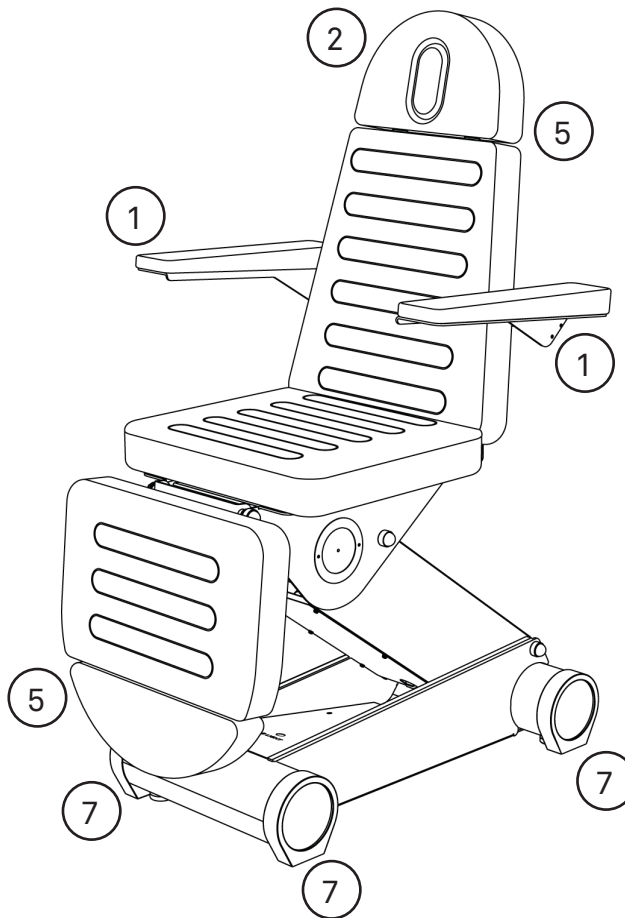

## Operation BLight 502

### Hand controller


The BLight 502 is equipped with 4 engines.

Use the backrest inclination keys to tilt the backrest up and down.

Use the legrest inclination keys to tilt the legrest up and down.

The height of the bed can be adjusted too. To lift the bed, simply push the UP button on the hand switch. Once the desired height is achieved, release the button. To lower it, do the same operation with the DOWN button. (Third row of keys)

You can also adjust the whole inclination of the bed downwards or upwards using the seat inclination keys.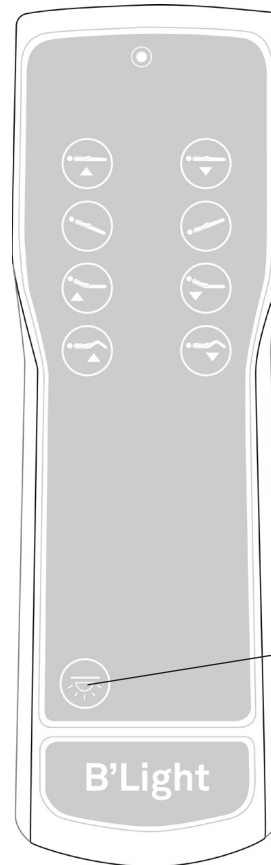

- Up & Down
- Seat inclination
- Backrest inclination
- Legrest inclination


ON/OFF LED Light

### Foot controller

A foot controller is available as an option and it can be ordered separately. You can adjust all the 4 movements of the bed by means of the foot controller.

In order to use both the hand controller and the foot controller simultaneously, you need a branched cable and you must plug it into the control box.

See instructions in picture below.

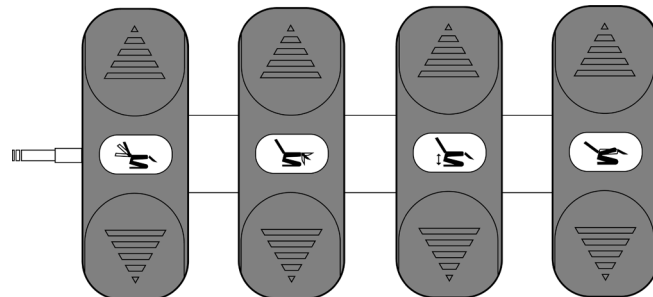


### Connecting the foot controller

Locate the control box inside the table base frame. Take the safety piece that locks the connectors into each plug and disconnect the cable HS.

Connect the branched cable in the plug named HS. Place again the safety piece to lock the connectors.

Now you can connect the hand controller and the foot controller in the two plugs.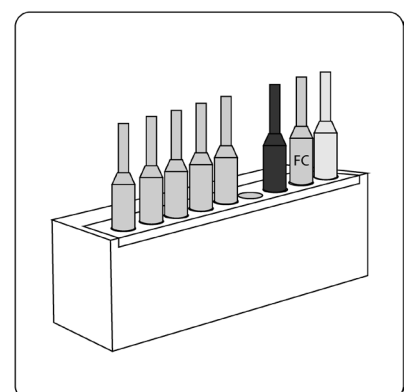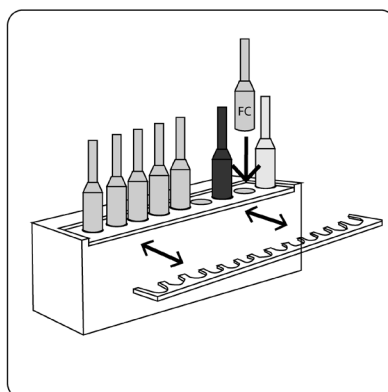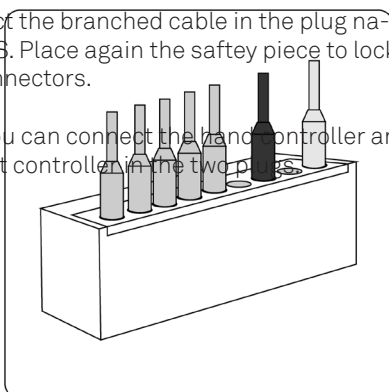

## Operation BLight 503


### Hand controller

The BLight 503 is equipped with 3 engines.

Use the backrest inclination keys to tilt the backrest up and down.

The height of the bed can be adjusted too. To lift the bed, simply push the UP button on the hand switch. Once the desired height is achieved, release the button. To lower it, do the same operation with the DOWN button. (Third row of keys)

You can also adjust the whole inclination of the bed downwards or upwards using the keys in the second row.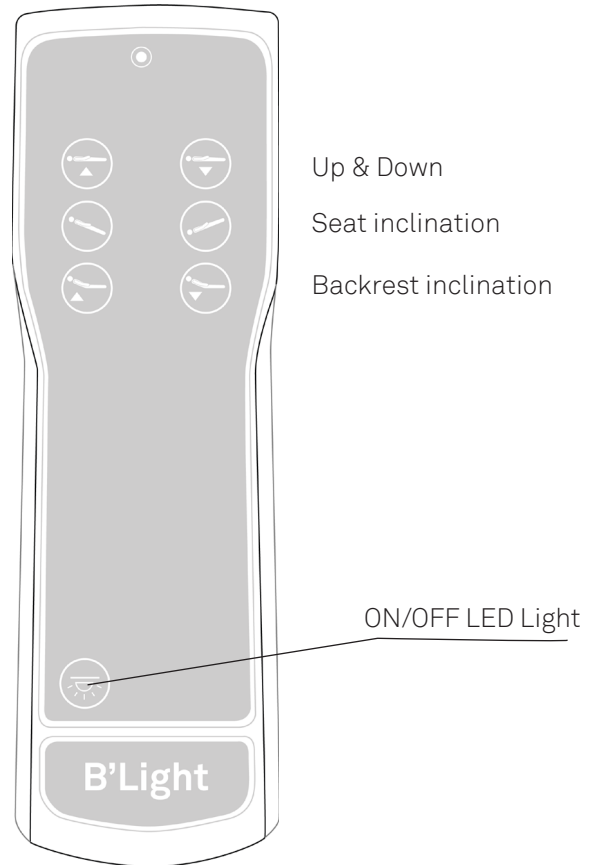


### Foot controller

A foot controller is available as an option and it can be ordered separately. You can adjust all the 3 movements of the bed by means of the foot controller.

In order to use both the hand controller and the foot controller simultaneously, you need a branched cable and you must plug it into the control box.

See instructions in picture below.

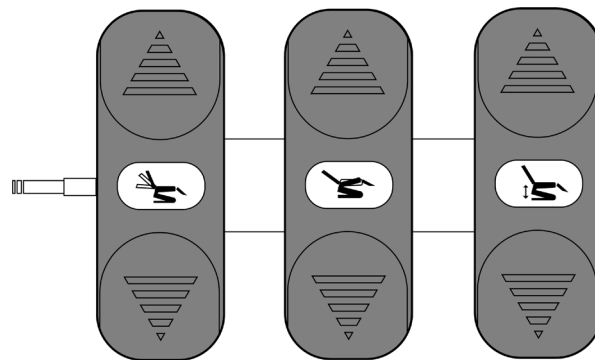


### Connecting the foot controller

Locate the control box inside the table base frame. Take the safety piece that locks the connectors into each plug and disconnect the cable HS.

Connect the branched cable in the plug named HS. Place again the safety piece to lock the connectors.

Now you can connect the hand controller and the foot controller in the two plugs.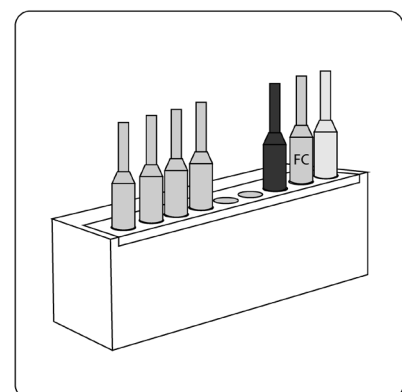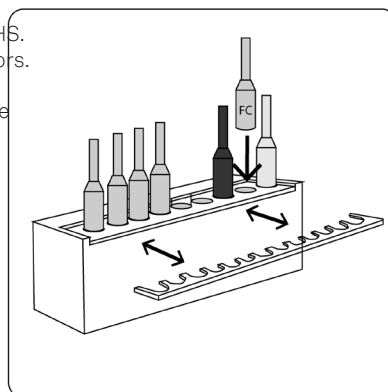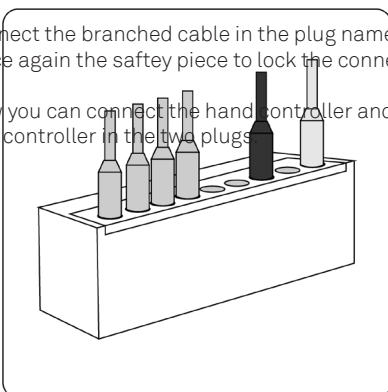


## Legs (BLight 503)

The BLight 503 podiatry bed is equipped with 2 leg rests that can be moved up and down individually to the vertical desired position.

To adjust them use the levers placed at each side of the leg rests to activate the gas lift. They can be placed up to horizontal plane (90°).

Extensible foot rests that can be adjusted according to the height of the patient.

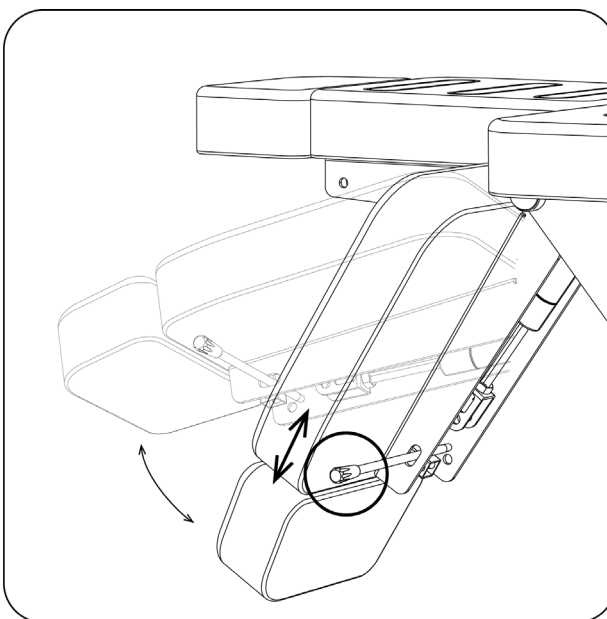## Specifications

Product size	178 (+/- 26 cm with movement) x 83 x 58 / 94 cm
Package size	127 x 62 x 61 cm
Highest position (flat)	94 cm
Highest position (trendelenburg)	121 cm
Lowest position	58 cm
Net weight	502: 91Kg / 503: 106Kg
Gross weight	502: 93Kg / 503: 108Kg
Power	AC 100 - 240V, 50/60Hz
Max power	300W
Environment temperature	0°C - +40°C
Relative humidity	30% - 90%
Max weight supported	175 Kg

## 8. Maintenance and cleaning

### Upholstery Daily Cleaning

It is very important for the life of the upholstery to clean it properly.

- 1.- Clean the entire surface using a mixture of 1:9 liquid soap (neutral pH) and water. Do not use soap or cleaning solutions that contain alcohol, ketones, xylene, acetates or solvents (mineral/white spirits).
- 2.- Completely remove excess cleaner with a white, clean, damp cloth.
- 3.- Dry the surface.

For tougher stains refer to Spradling cleaning guidelines for PERMABLOK3 & PERMAGUARD vinyl upholsteries.

- Important: Lacquers, strong cleaners, xylene-based solutions, acetone or MEK cause immediate damage and contribute to the deterioration of the material. The use of such cleaners is at the owner's risk.

### Table Frame Cleaning

Remove dust with a white cloth.

Use alcohol-free cleaning wipes to clean the structure or simply use a white cotton cloth soaked in a mix of water with neutral diluted soap and rinse. Dry with a clean cloth.

### Maintenance

Our tables have been designed for an intensive use during long time thanks to their stability, reliability and durability. However, we advise to keep the table in perfect conditions and make a correct use of it. Unplug the table if you will not use it for a long period of time.

### Environment information

This symbol indicates that some electrical components or materials can be reused and/or are harmful to the environment and thus, they must be properly recycled. Ensure that you are familiar with the local rules and regulations regarding to the disposal of equipment and accessories.

## 9. Troubleshooting

Squeaking noise when moving the bed up & down (mechanical noise):

- The axes are adjusted and greased from factory but with the time and use, they may need some little adjustment. Remove the grey plugs of the connecting rods (2 at each side, so 4 in total). Loosen the axes a little bit by unscrewing them (approx. 2 turns). Apply a bit of neutral grease and plug them again.

None of the motors work:

- Check if the power cable is disconnected, either from the wall socket or the control box.
- Check the power cable for any damage.
- Check all the plugs and connections.
- Check if the LED light in the control box is lightened GREEN.

If 1 motor is faulty:

- Follow the TEST to determine whether the source of the problem lies in the control box, the hand controller or motor of the affected part. Exchange the cable of the faulty motor with another section that works properly to discard.

GET access to the control box. The control box has the connections of the hand controller, the 4 engines of the bed and the power supply cable.

If the motor of the backrest does not work, unplug for example the HEIGHT motor and plug this wire in the BACKREST motor socket (exchange socket from other motor that works fine). Check the movement of the HEIGHT motor by pressing the buttons of the BACKREST movement in the hand controller to observe the motor operation.

- If the motor does not work, the problem is the motor.
- If the motor works properly, the problem would be the control box.

### Persisting problems

If problems persist, please contact your dealer and provide the serial number and model of the table as they appear under the bed.

## 10. EMC Guidance

Guidance and Manufacturer's Declaration - Electromagnetic Emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class A
Harmonic emissions IEC 61000-3-2	Class A
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies

Warning: It is not safe to operate the device in the presence of HF surgical equipment or places where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this device and other equipment should be observed to verify that they are operating normally.

Warning: Use only accessories and cables provided by the manufacturer. Accessories and cables not recommended may affect the EM emissions or immunity.

Warning: Maintain minimum distance of 12 inches (30 cm) between any part of the working table and portable RF communication equipment (including peripherals such as antenna cables and external antennas.) Performance of the table might degrade if proper distance is not maintained.

ble RF communication equipment (including peripherals such as antenna cables and external antennas.) Performance of the table might degrade if proper distance is not maintained.

Guidance and Manufacturer's Declaration - Electromagnetic Immunity		
Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±kV, ±15 kV air
Electrical fast transient /burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines
Surge IEC61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0% U; 0.5 cycle T at 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 0% UT; 1 cycle 70% UT; 25/30 cycle 0% U; 250/300 cycle	0% U; 0.5 cycle T at 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 0% UT; 1 cycle 70% UT; 25/30 cycle 0% U; 250/300 cycle
Power frequency (50/60 Hz) Magnetic field IEC61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	3 V r.m.s. 150 kHz to 80 MHz 6 V RMS in the ISM bands between 0.15 MHz and 80 MHz	3 V r.m.s. 150 kHz to 80 MHz 6 V RMS in the ISM bands between 0.15 MHz and 80 MHz
Radiated RF IEC61000-4-3	3 V/m 80 MHz - 2,7 GHz 80% AM at 1 kHz	3 V/m 80 MHz - 2,7 GHz 80% AM at 1 kHz
NOTE: UT is the a.c. mains voltage prior to application of the test level.		

Test Specifications for enclosure port immunity to RF wireless communications equipment					
Immunity Test	IEC60601 test level				Compliance level
	Test frequency	Modulation	Maximum power	Immunity level	
Radiated RF IEC 61000-4-3	385 MHz	**Pulse Modulation: 18Hz	1.8W	27 V/m	27 V/m
	450 MHz	*FM+ 5Hz derivation: 1kHz sine	2W	28 V/m	28 V/m
	710 MHz 745 MHz 780 MHz	**Pulse Modulation: 217Hz	0.2W	9 V/m	9 V/m
	810 MHz 870 MHz 930 MHz	**Pulse Modulation: 18Hz	2W	28 V/m	28 V/m
	1720 MHz 1845 MHz 1970 MHz	**Pulse Modulation: 217Hz	2W	28 V/m	28 V/m
	2450 MHz	**Pulse Modulation: 217Hz	2W	28 V/m	28 V/m
	5240 MHz 5500 MHz 5785 MHz	**Pulse Modulation: 217Hz	0.2W	9 V/m	9 V/m
<p>Note* - As an alternative to FM modulation, 50% pulse modulation at 18 Hz may be used even if it does not represent actual modulation.</p> <p>Note** - The carrier shall be modulated using a 50% duty cycle square wave signal.</p>					

! Our products are subject to continuous technical improvements. Weelko reserves the right to change or modify the product and contents without prior notice.

Pol. Ind. La Valldan  
Camí de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com

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