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 **IMPORTANT:** Supplement to Maintenance section of all HeartSine® samaritan® PAD user manuals/ user handbooks

 **IMPORTANT:** Supplement to Maintenance section of all HeartSine samaritan PAD user manuals

Please store this supplement with your user manual/user handbook and AED.

As part of the monthly maintenance check, HeartSine Technologies recommends that the user performs the following check once every three months. This can be done quickly, without removing the AED from its case.

Press the On/Off button to turn on the AED

Listen for, but do not follow, the voice prompts to ensure you can hear the prompts If you do not hear a prompt, contact your authorized distributor or HeartSine Technologies directly

If you hear the prompt “Adult patient” and/or “Call for medical assistance,” no further action is needed

Press the On/Off button again to turn off the AED and verify that the status indicator is flashing green

If you hear a warning message or see a red flashing status indicator, consult Troubleshooting in the user manual/user handbook

You can record the AED serial number (SN) and year (YYYY), month (MM) and day (DD) for each of these maintenance checks in the log provided at the end of this supplement for your convenience

SN _____

YYYY	MM/DD	YYYY	MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
YYYY	MM/DD	YYYY	MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
YYYY	MM/DD	YYYY	MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
YYYY	MM/DD	YYYY	MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD
	MM/DD		MM/DD

heartsine.com

HeartSine Technologies, Ltd.
 207 Airport Road West
 Belfast
 Northern Ireland
 BT3 9ED
 United Kingdom
 Tel +44 28 9093 9400
 heartsinesupport@stryker.com
 heartsine.com

Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: HeartSine, samaritan, Stryker. All other trademarks are trademarks of their respective owners or holders.

Made in U.K.

M0000019056-AA

Copyright © 2024 Stryker.